



Installation description –

How to mount a 2- or 4-point belt to a chair having a standard seat plate

- 2-point belt for a chair having standard seat plate - you need kit 45279
- 4-point belt for a chair having standard seat plate - you need kit 45280

Item Kit = 45279 consist of:		
Amount	Item number	Description
2	86282	Universal Fastener Kit
2	41698	Belt Adapter
2	20032	Bolt M8
2	20068	Washer M8
2	20412	Lock nut M8

Item Kit 45280 consists of 2 X 45279

To mount the Universal Fastener Kit to a chair having a standard seat plate, please follow the instruction MD0109UK you can download from our homepage:

<http://www.my-netti.com/docs/mounting-descriptions/mounting-description-belt-bracket-kit.pdf?sfvrsn=6>



As soon you have installed the Universal fastener kit, you can mount any 2-point belt to the L-bracket.

With kit 45279 you get two belt adapter 41698.

You can add them to the L-Bracket together with the washer, bolt and lock nut like shown in the below pictures.

For a 4-point belt, you need four belt adapter. Please order the kit 45280.

The installation of the belt adapter always remains the same.



Please follow the instruction on the next page to adjust the belt.



Two-Point Pelvic Support Belt



1. Locate desired anchor point.
2. Center the belt on the pelvis and thread the webbing end through the Flat-Mount bracket.
3. Cinch the webbing snug.

Four-Point Pelvic Support Belt



1. Locate desired anchor point
2. Follow the installation instructions above to fit the primary straps.
3. Attach the secondary straps to the chair so that they anchor the primary straps.
4. Tighten secondary straps to prevent movement of the primary straps.



ADJUSTMENT:

When properly adjusted and the belt tightened, it should fit snug so that the user's pelvis is secure. On padded belts, tighten the D-ring pull until all slack is removed and the pads under the buckle are touching. If length adjustment is necessary to achieve a good fit or to remove the belt for cleaning, this can be done using the belt end fittings.



SAFETY CHECK:

When properly positioned in a wheelchair, have user lean forward and side to side to check fit.

Check for:

1. Normal operation of buckle and adjustment straps.
2. Comfort - Look for areas of irritation.
3. Position -If too high or too low. Adjust anchor points.
4. Interference with other devices - Relocate anchor points as necessary.

This instruction is part of every 2- or 4-point belt and must, therefore, be forwarded to the user or carer and should be stored.

If you have any further question related to this item or mounting description, please contact our customer service desk.

This mountings description and others are available for download from our homepage:

<http://www.my-netti.com/downloads/manuals>