



## Mounting description – Head support stabiliser (D23017)



1. Remove the screw from the push bar brackets on both sides.  
(Attention, fix the nut inside of the tube with a tong).  
If you are using back height 60 cm, please replace the original screws with longer screws ( M6 X 25 mm )



2. Remove the 4 screws from the head support adaptor.



3. Now place the head support stabiliser over the push bar brackets and bring the 4 holes in the same position as the 4 holes of the head support adaptor.



4. Fix the head support stabiliser with the original screws at the push bar bracket.  
(Attention, you don't tighten the screws too tightly yet)  
If you are using back height 60 cm, please replace the original screws with longer screws ( M6 X 25 mm ).



5. Now put the 4 enclosed socket-head cap screw ( 50 mm long ) into the bore-holes and fix the head support stabiliser with the head support adaptor.



6. Now tighten all the screws firmly.

If you have any further question related to this item or mounting description, please contact our customer service desk.

This mountings description and others are available as a download from our homepage:  
<http://www.my-netti.com/downloads/manuals>