





Mounting description Side support Stabil

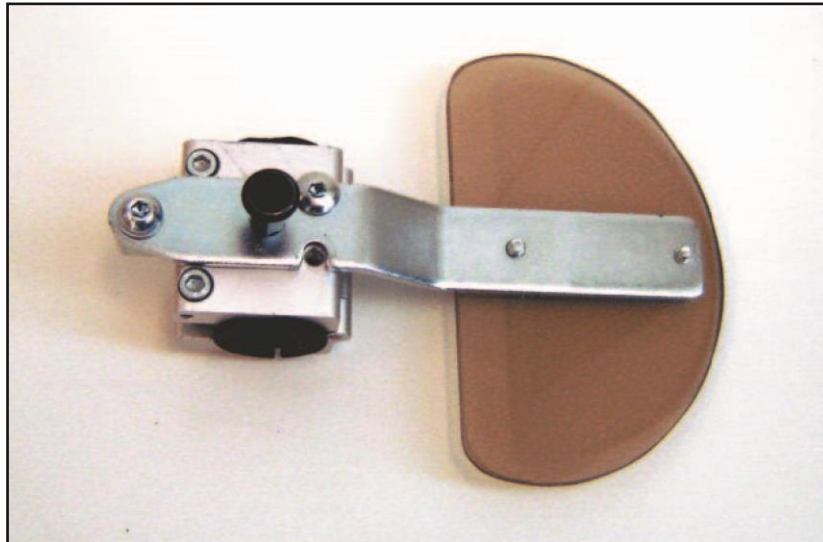

Item	Item number
Side support Stabil Left	26830
Side support Stabil Right	26843

Tools: 6 mm Allen key and a sharp knife

Remove the 2 screws holding the side support clamp and open the clamp.

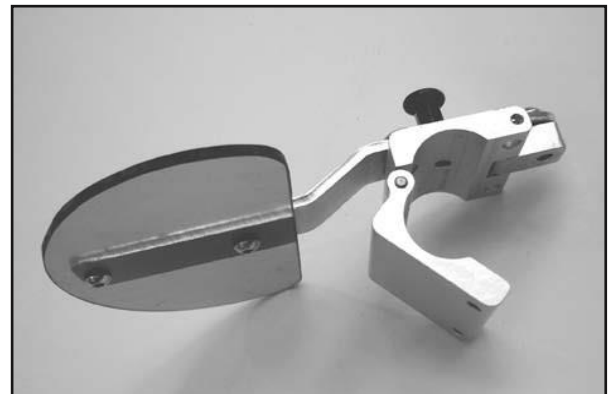


If the side support is being mounted to a wheelchair with square backrest tubes, please use the enclosed plastic bushing.





After fitting the backrest cushion, find the correct height for the side support. The support should be parallel with the integrated side support of the backrest cushion. Cut a piece of the Velcro backrest to give room for the clamp to fit around the backrest tube. When mounting on a square tube, place the enclosed plastic bushing around the backrest tube.

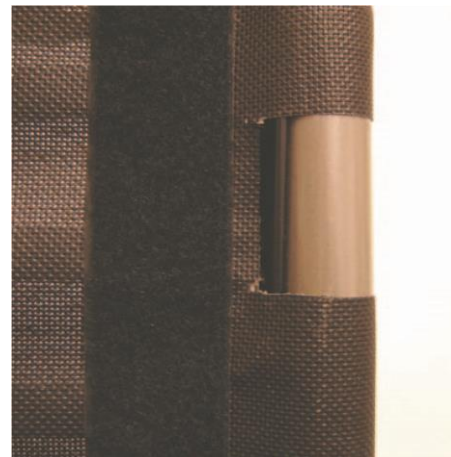

Mount the clamp and tighten the 6mm screws firmly. Before the screws are tightened, you can choose the angle of the clamp and thereby increase or decrease the stability.



To release the side support, pull the release button out.

