

# MOUNTING DESCRIPTION

## Y-belt

Item no 42208:

- Two brackets
- Two bolts
- Two discs
- Belt with straps

1.  
Mount the brackets to each side of the frame using the discs, bolts and nuts



1

2.  
The widest straps is to be fixed in these brackets



2

3.  
Assemble the strap. Make sure the strap is properly fixed and self locking by using the buckle on the strap



3

4.  
The narrow strap is to be fixed to the frame in front of the arm rest bracket. Also here the straps must be fixed and made self locking



4

5.  
Fit the belt to the person. Make sure it is comfortable and not too tight



5

6.  
The belt close and open by operating the buckle in front



6

If the strap ends after adjusting is too long just cut it and heat the cut to prevent unraveling