

MOUNTING DESCRIPTION

ABDUCTION-BLOCK WITH STABILIZER-KIT

86096 - Abduction block small

86097 - Abduction block medium

86098 - Abduction block



Enable joy of life

NETTI ABDUCTION BLOCK + STABILIZER KIT

The abduction block will be always delivered together with a stabilizer kit to reinforce the seat plate.

Please mount first the stabilizer kit and after the fastening bracket of the abduction block.

You need following tools: Awl, drilling machine, 6mm drill, 4mm Allen key and open-end spanner size 10mm

The stabilizer kit contains:



Number in drawing Description Quantity:

1	Metal plate	1
2	Square washer	2
3	M6 lock nut	2
4	Hex head screw M6x25	2
5	Hex head screw M6x22	2

The metal plate (1) can be used for all kinds of seating plates except of Netti 4U FA and Netti Dynamic System chairs.

MOUNTING DESCRIPTION STABILIZER KIT:

1. Place the metal plate (1) in centre of the seating plate like shown in the drawing
2. Use an awl to mark the position of screw holes
3. Drill 2 holes for hex screw M6x25 (4)
4. Drill 2 holes for hex screw M6x22 (5)
5. Place the washer from the underside and put the hex head screws into the drilled holes
6. Fix the M6X22 with the 2 locking nuts M6 by using a 4mm Allen key and an open-end spanner size 10
7. The hex head screws M6x25 (4) will be used to fix the fastening bracket.

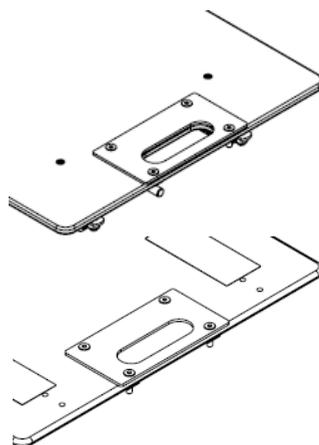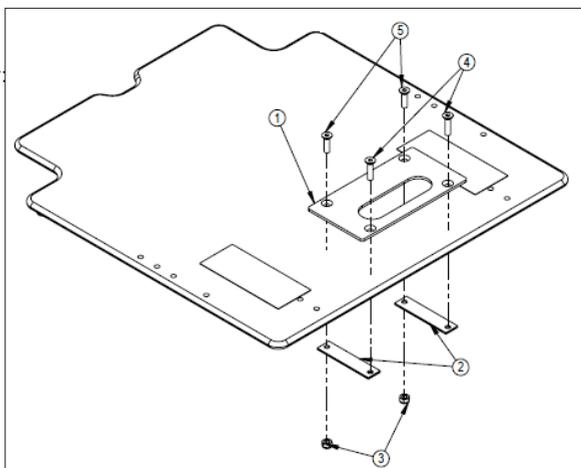

MOUNTING DESCRIPTION ABDUCTION BLOCK:

The abduction block comes with a fastening bracket.



1. Place the fastening bracket at the underside and in the center of the seat plate on top of the washer. Fix the locking bracket and the metal plate with 2 M6x25 hex head screws.

2. Place the bar for height adjustment of the abduction block between the fastening bracket and the red star wheel.

Fix the abduction block by turning the red star wheel clockwise.



3. The abduction block can be adjusted 80mm in height (B) to be compatible with all Netti seating cushions and others.

The abduction block can be adjusted 100mm in depth (A) for an individual fitting.



The abduction block is available in 3 sizes:

Small 80x140x80mm


Medium 110x150x80mm

Large 140x170x80mm

The adduction block can not substitute any pelvis stabilization belt!

Individual sizes can be ordered on request as special adaptation.

This manual is also found on our homepage www.My-Netti.com.

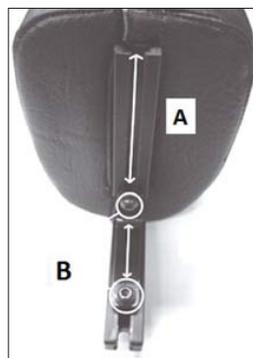

IN DIALOGUE WE CREATE
SIMPLE SOLUTIONS AND
ENABLE JOY OF LIFE



This product conforms to
93/42/EEC for medical products.

MD0115 UK
2015-11

FEEL FREE TO CONTACT US:



Alu Rehab AS
Bedriftsvegen 23
4353 Klepp Stasjon
NORWAY

T: +47 51 78 62 20
post@My-Netti.com

Alu Rehab ApS
Kløftehøj 8
DK-8680 Ry

T: +45 87 88 73 00
F: +45 87 88 73 19
info@My-Netti.com

My-Netti.com