

MOUNTING DESCRIPTION

FOOT BOX AND
LOCKING BRACKET




Enable joy of life

MOUNTING DESCRIPTION FOOT BOX

The foot box for your ordered seat width is delivered with foot rest tubes and 2 distance clips.

Number in drawing	Description	Quantity
1	Foot rest tubes	1
2	Foot box	1
3	Distance-Clips	2

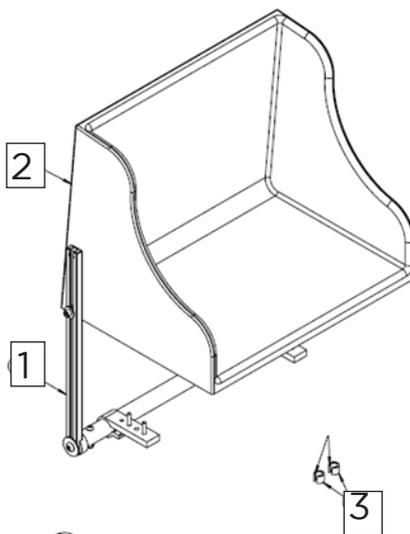


1. On both leg supports loosen and remove the bolts which are used to adjust the foot plates heights (A). Pull the foot rest tubes with foot plates downwards and out of the leg support.

Remove calf supports and their brackets.

2. Insert the foot rest tubes (1) of the foot box (2) into the leg support (B) and tighten the bolts again.

Finally adjust the foot box height according to your need.

3. Put the distance clips (left and right) on the silver coloured rail of the leg support (C). This provides a certain distance to the front wheels preventing collision with them.

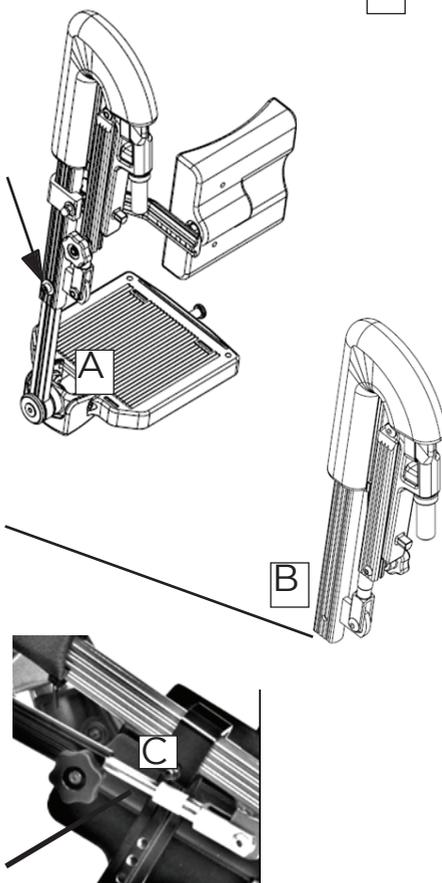

EXCHANGE OF LOCKING BRACKETS WITHOUT STOP (OPTIONAL)

As an optional solution you can have locking brackets without stop. With these mounted you can more easily remove the complete foot box from the chair.

As standard the locking brackets has a stop to avoid them from unintentionally slipping out of the locking brackets.

1. On both sides loosen the M6 Hex socket bolts from the M6 nuts and pull the bolts out sideways.

Pull the M6 nuts on both sides out of the slots.

2. Replace the locking brackets with stop, with the locking brackets without stop and place the M6 nuts back into the slots of the replaced locking brackets.

3. Put the screws back through frame part and the locking bracket and tighten the screws.



4. You are now free to mount the foot box to the chair by leading the cone into the locking bracket.

! A wheelchair with this foot box is not to be used as a seat in a car.

You can find this manual also on our web page:

<http://www.my-netti.com/downloads/manuals>

Feel free to contact us if you have further questions.

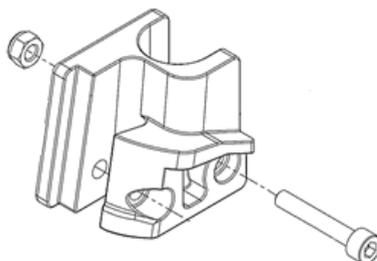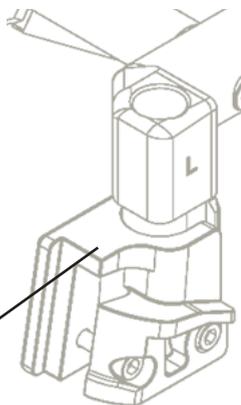

IN DIALOGUE WE CREATE SIMPLE SOLUTIONS AND ENABLE JOY OF LIFE

This mounting description is available in different languages to be downloaded from our homepage www.My.Netti.com

 This product conforms to 93/42/EEC for medical products.

MD0111UK
2018-12

FEEL FREE TO CONTACT US:



Alu Rehab AS
Bedriftsvegen 23
4353 Klepp Stasjon
NORWAY

T: +47 51 78 62 20
post@My-Netti.com

Alu Rehab ApS
Kløftehøj 8
DK-8680 Ry

T: +45 87 88 73 00
F: +45 87 88 73 19
info@My-Netti.com

My-Netti.com