



MOUNTING DESCRIPTION

Side support Basic for Netti 4U Base

Item number: 88468 – Can be used for left and right side

Tools:

Allen wrench 4 + 3

The side support for Netti 4U Base consists of 2 components:

1. a clamp for mounting to backrest tube
2. the side support (Type Basic)

The side support is adjustable in height, width and depth.

First, separate the 2 clamp parts by disassembling the 2 screws with a Allen wrench No. 4. After, put both parts around the backrest tube back again. Tighten the screws softly.

The backrest tube is now located between the two clamp parts.

Then position the clamp in the area shown on the right.

Adjust the clamp at the desired height and tighten the screws so that the clamp can be positioned later.

Now move the clamp lever at the clamp by turning the lever counter-clockwise until you touch the backrest tube, then pull the lever upwards and proceed to move the lever.

Repeat this until the hole in the clamp is clear.

You may also need to unscrew the grub screw on the bottom of the clamp by using the 3mm Allen wrench.

Now add the side support to the clamp and turn the lever clockwise to fix the side support in the clamp.






As soon as the user of the wheelchair sits in the wheelchair, position the side support in the desired height and width.

You can now also adjust the angle of the side support on the backrest tube, by moving the side support inwards or outwards.

As soon as you have finished the adjustment, tighten the 2 screws firmly.

The position of the clamp can be secured with the 2 at the front of the clamp additional grub screws.


To prevent unauthorized removal of the side support, you can tighten the screw opposite of the clamp lever with an No. 3 Allen wrench.

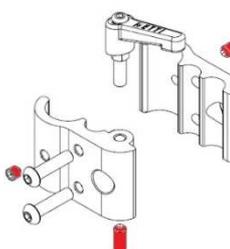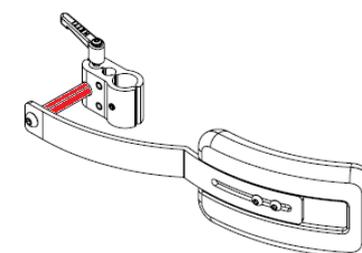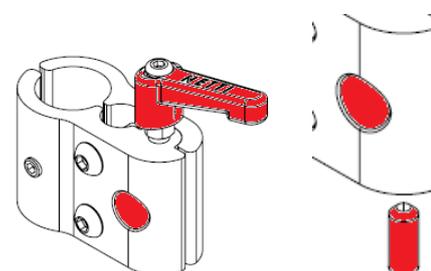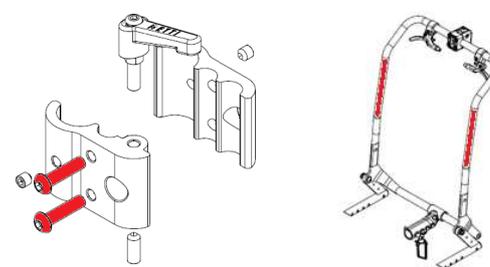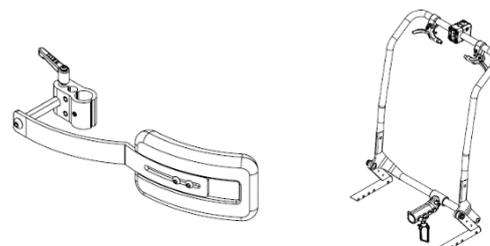




To adjust the side support in depth, loose the 2 screws at the backside of the side support pad and move the pad to the desired depth.

Tighten the 2 screws after.

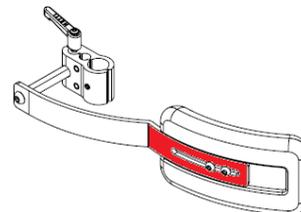

You can use the side support for Netti BASE on the left or the right side.

To use the pad for the left or right side, turn the clamp with 180° and mount the clamp lever on top and the grub screw at the bottom.

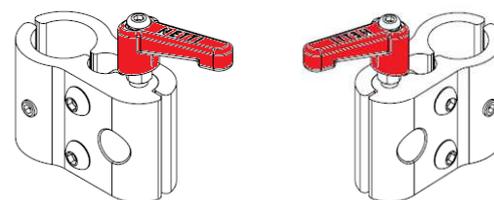

This instruction is part of every side support and must therefore be forwarded to the user or carer and should be stored.

If you have any further question related to this item or mounting description, please contact our customer service desk.

This mountings description is also available as download from: <http://www.my-netti.com/downloads/manuals>