

RE-PROGRAMMING IF
DX2 MASTER REMOTE OR
DX2 POWER MODULE IS
EXCHANGED

FOR NETTI MOBILE

Enable joy of life

NETTI MOBILE

WHAT HAPPENS IF YOU EXCHANGE A DX2 MASTER REMOTE OR DX2 POWER MODULE?



DX2 SYSTEM BACKUP MIRROR

DX2 has a System backup Mirror. It works only if the control system has a DX2 master remote and a DX2 power module.

A copy of the wheelchair program is stored in the Power Module.

This means you can exchange the DX2 master remote or the DX2 power module without having to reprogram the control system.



DX2 REM420



DX2 Power module

DX2-REM420

If the control system detects that the program in the DX2 remote is different from the program in the DX2 power module, actuator LED's 1 and 4 flash simultaneously.

You must choose between the program in the remote and the program in the power module.

To select the program in the power module, press the actuator minus button - LED 1 flashes.

To select the program in the master remote, press the actuator plus button - LED 4 flashes.

To confirm your choice: press the speed + button and the horn together for 3 seconds.

This causes the system to turn-off and re-start.



If the wheelchair has a special program and you need to replace both units (remote and the power module) you will need to save the existing program using Dynamic Wizard to a PC. After installing both modules you will need to write the original program back to the chair.

For optimal customer service we recommend that a copy is kept whenever a new non-standard program is created and labeled with the wheelchair serial number.



IN DIALOGUE WE CREATE
SIMPLE SOLUTIONS AND
ENABLE JOY OF LIFE



This product conforms to
93/42/EEC for medical products.

MD0103 UK
2016-03

FEEL FREE TO CONTACT US:



Alu Rehab AS
Bedriftsvegen 23
4353 Klepp Stasjon
NORWAY

T: +47 51 78 62 20
post@My-Netti.com

Alu Rehab ApS
Kløftehøj 8
DK-8680 Ry

T: +45 87 88 73 00
F: +45 87 88 73 19
info@My-Netti.com

My-Netti.com