MULTI LANGUAGE UK, DE, FR, SE

MOUNTING DESCRIPTION

FRONT CASTOR TOP-DOWN
QUICK RELEASE AXLE

Enable joy of life

The top down quick release axle fits to all Netti wheelchair front castor bearing houses.

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UK: Remove the top cap of the bearing house (which normally is mounted to the wheelchair frame).

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UK: Push the top button of the top-down quick release axle and insert it into the bearing house from the top.

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UK: Press the new soft top cap on to the bearing house.

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UK: Press the top button of the quick release through the soft top cap and insert the quick release in to the front castor fork.

Pull the front castor downwards to check that the quick release axle is firmly fixed to the fork.

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UK: The front castor is removed by pushing the top button of the Quick release through the soft top cap and pull the front castor and fork down. The axle still remains in the bearing house and sticks out downwards.

Remove the soft cap and the quick release axle from the bearing house. Place it in the front castor fork for transport. Make sure to keep the soft cap and replace it when the front castor is mounted again.

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IN DIALOGUE WE CREATE SIMPLE SOLUTIONS AND ENABLE JOY OF LIFE

This mounting description is available to be
downloaded from our homepage www.My-Netti.com

This product conforms to
93/42/EEC for medical products.

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