



Installation instruction

Body Point Ankle Huggers

Version 01-2025

for Netti III family, Netti V, Adapt pro, 4U family




MD0110 UK 2025-01

NORWAY: Bedriftsvegen 23 | N-4353 Klepp stasjon | post.klepp@meyragroup.com | Phone +47 51 78 62 20
DENMARK: Alu Rehab ApS | Kløftehøj 8 | DK-8680 Ry | info.ry@meyragroup.com | Phone +45 87887300

NEW 2018 DESIGN

Ankle Huggers®

The original, patented solution for stabilizing the lower legs without immobilizing the feet. Updated design features the low-profile ErgoLatch™ buckle plus easy hook-and-loop adjustment for the right fit every time.

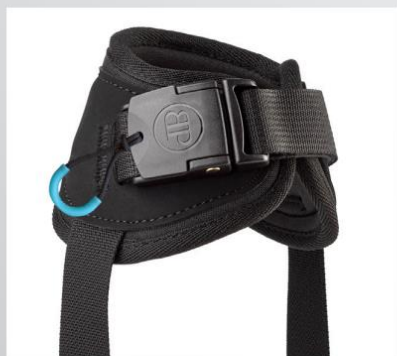

Part Numbers

Part Number	Fits Ankle Circumference
FT240XS	14–17cm (5-1/2–6-3/4")
FT240S	16.5–20cm (6-1/2– 8")
FT240M	19–25.5cm (7-1/2– 9-1/4")
FT240L	23–28cm (9– 11")
FT240XL	26.5–33cm (10-1/2– 13")

Replaces FT210 and FT220 styles

Order, quote, track shipments at bodypoint.com
206.405.4555 or 800.547.5716 | sales@bodypoint.com | Fax: 206.405.4556

© 2020 Bodypoint. All rights reserved. BMM304 2020.4



MD0110 UK 2025-01

NORWAY: Bedriftsvegen 23 | N-4353 Klepp stasjon | post.klepp@meyragroup.com | Phone +47 51 78 62 20
DENMARK: Alu Rehab ApS | Kløftehøj 8 | DK-8680 Ry | info.ry@meyragroup.com | Phone +45 87887300

Mounting description – Bodypoint Ankle Huggers

Together with the Ankle Huggers from Bodypoint you get the Netti-Mounting kit 45024 to connect Ankle Hugger Strap Positioner to a Netti-Foot plate.

The mounting kit 45024 contains of 8 bolts M5X16, 8 washers M6 and 8 lock nuts M5. 4 bolts and accessories per foot.

1. Find the required position for the feet on the foot plate and use one strap positioner per side.
Mark the hole of the strap positioner and drill 2 holes per strap positioner into the foot plate.



2. Plug the bolt into the holes and add from underneath washer and lock nuts. Tighten the nuts firmly.



3. Repeat this at the other side
4. Now mount the ankle huggers to the strap positioner like described in the Bodypoint mounting description.



The Bodypoint description you will find on page 2 of this MD.

MD0110 UK 2025-01

Ankle Huggers mounted on Grandis foot plates

The ankle huggers are mounted to the foot plates, fixing the ankle hugger brackets 88658 on the underside of the foot plate - in the profile slots.

Mounting ankle hugger brackets to the foot plate: Insert square nut into slot and fix the bracket as shown:



MD0110 UK 2025-01