

MOUNTING DESCRIPTION

Backrest extender kit

Item no : 43799

Back extension kit for Netti III, Netti 4U CE plus and Netti 4U CED


1. Remove the plastic caps on top of the back tubes

2. By Netti III og Netti 4U Ce Plus:
Using an Allen key loosen the M6 screws **A** on each side as shown on picture 2. NB: Hold the nut inside with a pleyer, and remove it when the screw is loose.

On all models: Remove the screw **B** which is holding the back fabric on each back tube.



1

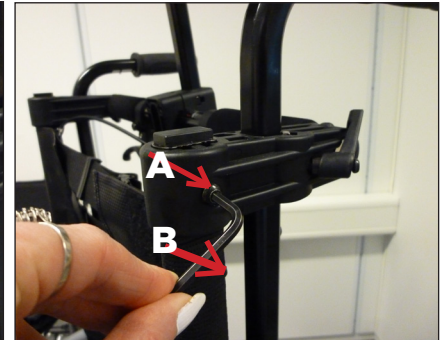

2

3. The back extender kit contains two extender tubes with each one piece Velcro.

4. Slide the extension pieces into the back tubes. The square nut inside the extension tube must face outwards.



3



4

5. Netti III og Netti 4U CE Plus:
Insert the M6 screw **A** into the push bow bracket and into the square nut sitting inside the extension bracket tube - on each side. Leave loose for the extension bracket to move up and down.

On all models: Insert the new M4 pan head screw **B** into the backwards turning hole from screw **B** in the back tube, and move the extension tube up and down till you find the desired threaded hole in the extension tube. There are 4 available height positions: +100 mm, +172,5 mm, +199 mm, +225 mm.

Tighten both sides M6 A-screws and the M4 B-back-screws.



5

Please note that the extended back may contribute to making the wheelchair more unstable when the seat unit and or the back are tilted backwards. Check the wheelchair stability with the user in the wheelchair after having installed back extender. Drive wheel position and anti-tippers may have to be moved backwards to keep a good stability.

