

# Mounting description

**UNIVERSAL belt fastener-kit 86282**

**Page 2-3:**

**Mounting a standard hip belt/  
2-point pelvic support or 4-point pelvic support**

**Page 4-5:**

**Mounting an Evoflex-Pelvic support from  
Bodypoint**


*Enable joy of life*

## Mounting a standard hip belt/2-Point belt or 4- Point belt

The Universal belt fastener kit 86282 consist of the lower shown parts + 1 L-bracket per side:



1. To get an easy access to the side frame, remove the seat plate.  
If the chair has a sliding seat this is maybe not necessary .

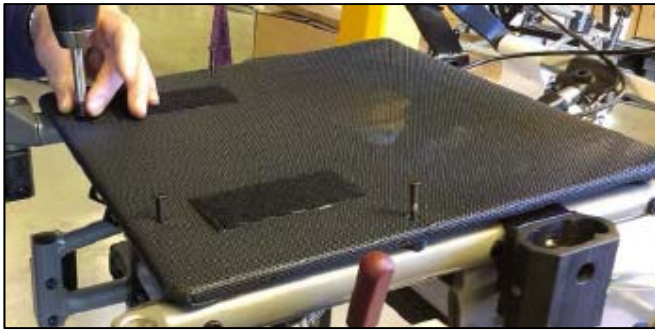

2. Add the clamp to the upper frame tube like shown below and add the bar spacer in-between the clamp.  
Be aware to have the square hole showing towards outside.



3. Now you have to add the carriage bolt from outside into the outer clamp part and the bar spacer. The long nut you have to add from inside into the inner clamp part and bar spacer.



4. Move the clamp like shown in the picture to the desired position at the tube and fasten the nut with a wrench.

5. Now you have to add the L-bracket on top of the clamp. Fasten the cylinder head screw firmly.



Now you can mount any belt to the L-bracket.

When using an Alu Rehab Hip belt you get 2 mounting brackets with the belt.

You can add them to the L-Bracket together with the washers like shown in the next pictures..



6. Repeat the steps 1-5 on the other side of the frame.

**!!! Please avoid mounting this kit to a chair with Dynamic seat plate. The belt kit is static and will prevent the dynamic movement of the seat plate. It can cause squeezing hazard.**

On the next pages you will find a mounting description of a Bodypoint Evoflex pelvic support.


## Mounting a Bodypoint Evoflex pelvic support

Steps 1-3 are the same like seen on a standard hip belt.

4. Turn the clamp downwards. The head of the nut needs to show to outwards direction. Fasten the nut firmly with a wrench.



5. With the next step you will fix the Evoflex pelvic support with the cylinder head screw to the clamp. To avoid any conflict with the main wheels, please remember to cut the belt along the milled notch.

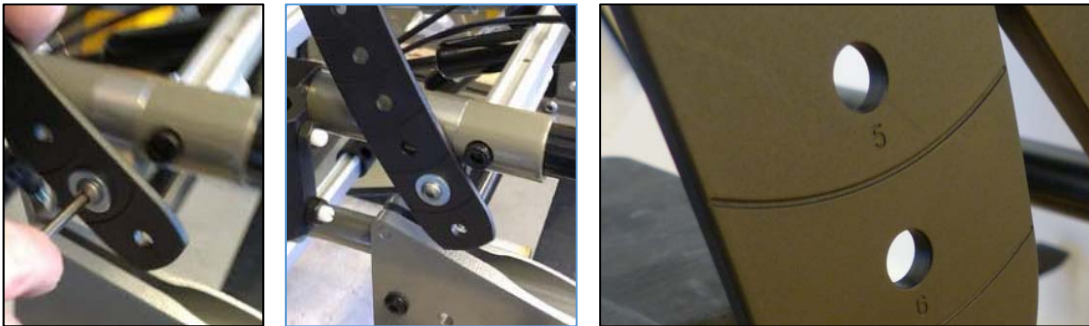

9. Repeat the steps 1-5 on the other side of the frame.

**!!! Please avoid mounting this kit to a chair with Dynamic seat plate. The belt kit is static and will prevent the dynamic movement of the seat plate. It can cause squeezing hazard.**

This Universal belt fastener Kit can be used for all standard belts and 2-point or 4-point pelvic supports like Evoflex belt from Bodypoint.


For a posterior pelvic tilt the 2-point belt can be used to improve the positioning when mounted at a 60°-90° angle to the seat.



Uncorrected posture

Improved positioning with  
two-point hip belt

For an anterior pelvic tilt the 4-point belt can be used to improve the positioning when mounted at a 30°-45° and a 60° - 90° angle to the seat.

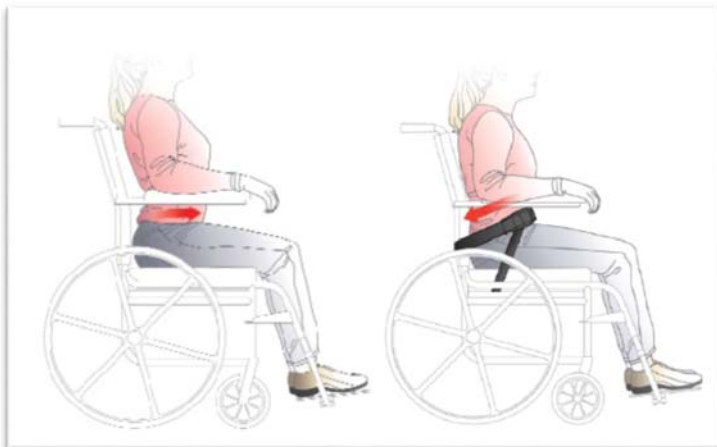

Uncorrected posture

Improved positioning with  
four-point hip belt

Mounting description  
Universal belt fastener kit 86282



IN DIALOGUE WE CREATE  
SIMPLE SOLUTIONS AND  
ENABLE JOY OF LIFE



This product conforms to  
93/42/EEC for medical products. 2015-12 UK

FEEL FREE TO CONTACT US:



Alu Rehab AS  
Bedriftsvegen 23  
4353 Klepp Stasjon  
NORWAY

T: +47 51 78 62 20  
post@My-Netti.com

My-Netti.com