

Netti KNEE- AND THIGH SUPPORT

Mounting Descriptions



inspire joy of life

MD0100UK 2024-11




KNEE- AND THIGH SUPPORT

You will get the knee-and thigh support always complete with mounting material.

The kit consist of:

Number in	Description	Quant.
drawing		
1	Attachment bar	1
2	Knee and thigh support	1
3	Hex socket screw M6x25	2
4	Washer M6	4
5	Lock nut M6	2

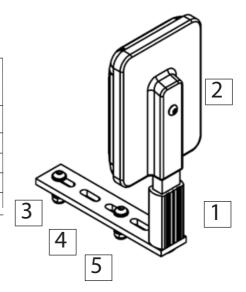

You need following tools:

Awl, drilling machine, 6mm drill, 4mm Allen key and open-end spanner size 10.

Please find mounting description on the next page.



- 1. Place the seat cushion on top of the seat plate.
- 2. Transfer the user into the chair and get him in the right position.
- 3. Place the attachment bar (1) at the left or right side of the thigh according to your need on top of the seat plate and under the seat cushion.
- 4. Find now the right support area at the thigh and mark this position.
- 5. Transfer the user out of the chair.
- 6. Mark the holes you are going to drill in the seat plate with an awl and drill 2 M6 holes (be aware of drilling the holes within the slots of the attachment bar to have the possibility to adjust the bar after)
- 7. Place 2 washer (4) on top of the attachment bar and put the hex socket screws (3) through attachment bar and seat plate.
- 8. Now put the lock nuts (5) with the other 2 washer (4) from the underside of the seat plate on the screws and tight them firmly.
- 9. Put the knee-and thigh support (2) into the attachment bar (1)
- 10. For a sideward transfer you can easily remove the knee-and thigh support by lifting the knee-and thight support upwards
- ! Please note to place the knee-and thigh support not to close to the thigh to avoid permanent high pressure!!!

If the wheelchair is used as a seat in the car, the knee and leg supports must be removed and stored elsewhere in the car.

You can find this manual also on our web page: http://www.my-netti.com/downloads/manuals

Feel free to contact us if you have further questions.



Manufacturer of Netti:

Alu Rehab AS

▶ Bedriftsvegen 23
N-4353 Klepp Stasjon
Norway

 post.klepp@meyragroup.com T: +47 51 78 62 20 my-netti.com

EU Distributor

Alu Rehab ApS Kløftehøj 8 DK-8680 Ry Denmark

info.ry@meyragroup.com T: +45 87 88 73 00 F: +45 87 88 73 19 my-netti.dk