




Assembling recline limiter to recline gas cylinder

The recline limiter kit consists of 3 different limiters (clips):

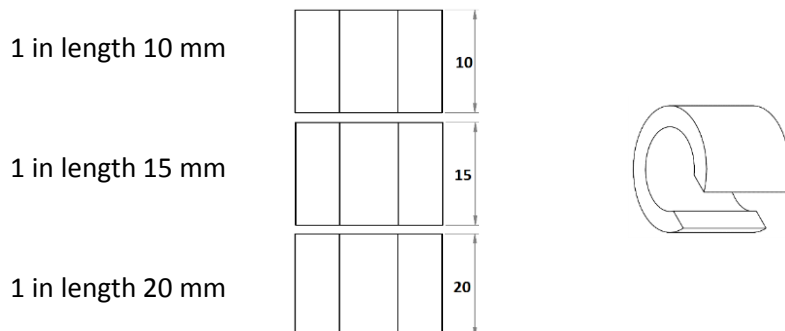

Assemble a single limiter or a combination of 2 or 3 limiters to the recline gas cylinder as shown in the picture to reduce the stroke range and achieve the desired back angle.



The matrix below helps you to find the desired back angle :

Recline limiter and recline limiter combination	Length	Back angle to achieve
No limiter	0 mm	134 °
A	10 mm	130°
B	15 mm	128°
A + B	25 mm	124°
C	20 mm	126°
A + C	30 mm	122°
B + C	35 mm	120°
A + B + C	45 mm	116°

MD0118UK