## MOUNTING DESCRIPTION NETTI THERAPY TABLES

### Netti SWINGABLE Tray - with lock

Chair width/ Sitzbreite	Description / Beschreibung	Item number / Artikelnummer	for chair models / für Stuhlmodelle
250 - 300 mm	swingable w lock	15139	Netti II/ III/ 4U CE/ 4U CES
350 - 400 mm	swingable w lock	15140	Netti II/ III/ 4U CE/ 4U CES
430 - 500 mm	swingable w lock	15141	Netti II/ III/ 4U CE/ 4U CES

**GB** The Netti swingable tray is easily swivelled to the side to make transfer and access easy for the user. The tray can be locked in the desired position.

#### Preparation for mounting the swingable tray

- The front screw (Pos. 2) that holds the arm support pad should be screwed out. When this is done, the wheelchair is ready for mounting of the tray. Mounting description is found on the next page.

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
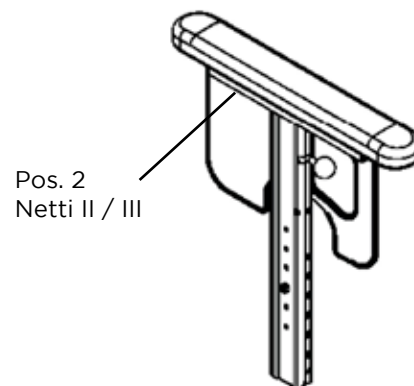
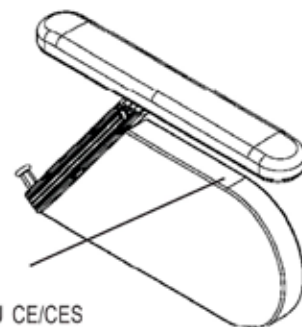
## GB The Netti swingable tray

### GB - Mounting description

- The holder (Pos. 1) with the star wheels shall be mounted in the low edge of the arm support. Use the rear hole and mount the arm support pad with machine screw M5 x 35. Do not use the washer.
- The tray can now be mounted on the wheelchair. The tubes shall be pushed into the holder (Pos. 1) under the arm support. The screws (Pos. 3) that hold the slide tube under the tray in front should not be fixed. Push the tray all the way in.
- To put the tray in the centre position you have to check the distance from the outer edge of the tray to the outside of the arm support pad. The distance must be the same on both sides.
- Check that the tray easily slides into the holder.
- **Please note:** If comfort pads are mounted on the arm support pad, a spacer on each side - article 93282 - must be mounted on position 3 and 4 with 2xM6 x 28 screws.
- The screws (Pos. 3) should be tightened when the tray is in the rear position. The tray is now ready for use.

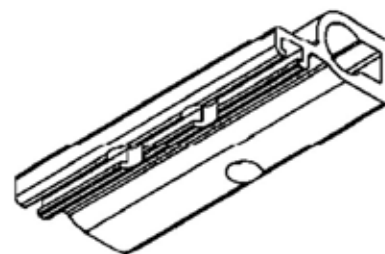*When adjusting to the widest option, the brackets must be slightly on the outside of the slide bar.*

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