MOUNTING DESCRIPTION NETTI THERAPY TABLES

Netti TRAY - WITH LOCK

Chair width/ Sitzbreite	Description / Beschreibung	Item number / Artikelnummer	for chair models / für Stuhlmodelle
350 - 430 mm	Tray with lock	28828	Netti 4U CED / 4U CEDS
450 - 500 mm	Tray with lock	28836	Netti 4U CED / 4U CEDS
350 - 430 mm	Swingable tray	28826	Netti 4U CED / 4U CEDS
450 - 500 mm	Swingable tray	28832	Netti 4U CED / 4U CEDS

NETTI TRAY - SWINGABLE

GB The Netti tray with lock; the locking permits safe positioning of the tray. It is reinforced and lockable.

o The Netti swingable tray is easily swivelled to the side to make transfer and access easy for the user. The tray can be locked in the desired position.

Preparation for mounting the lockable tray

o The front and rear screws (Pos. 2) that holds the arm support pad should be screwed out. When this is done, the wheelchair is ready for mounting of the tray. Mounting description is found on the next page.

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

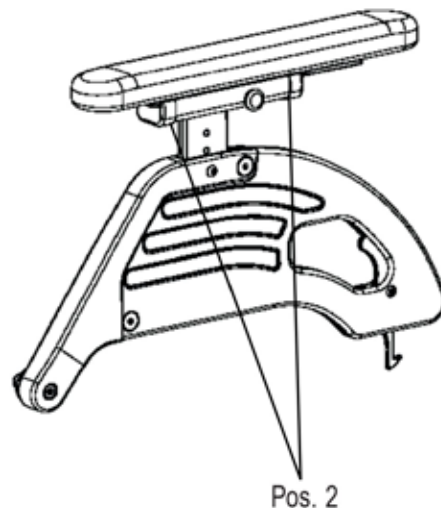
GB The Netti lockable tray and Netti swingable tray

Mounting description Tool: 3 mm Allen key

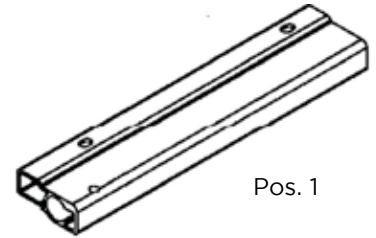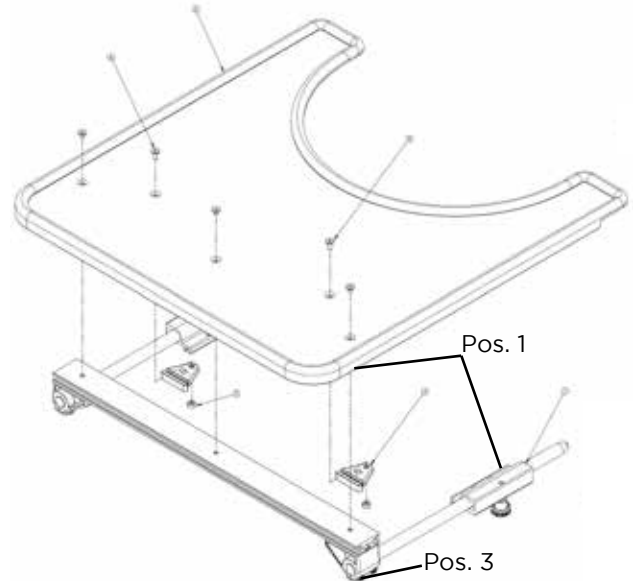- The holder (Pos. 1) with the star wheels shall be mounted between the arm support pad and the arm support profile. Insert the enclosed screws M5 x 33 through the arm support profile and holder and screw it into the arm support pad.
- The tray can now be mounted on the wheelchair. The tubes shall be pushed into the holder (Pos. 1) under the arm support. The screws (Pos. 3) that hold the slide tube under the tray in front should not be fixed. Push the tray all the way in.
- To put the tray in the centre position you have to check the distance from the outer edge of the tray to the outside of the arm support pad. The distance must be the same on both sides.
- Check that the tray easily slides into the holder.
- **Please note:** If comfort pads are mounted on the arm support pad, a spacer on each side - article 93282 - must be mounted on position 4 with 2xM6 x 28 screws.
- The screws (Pos. 3) should be tightened when the tray is in the rear position. The tray is now ready for use.

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