Netti[®] *Side support Stabil*

Mounting Description - uk

Montageanleitung - de

Monteringsanvisning - se

*inspire
joy of life*

NETTI SIDE SUPPORT STABILE (UK)

MOUNTING DESCRIPTION

Netti side support Stabile is developed to mainly give extra stability to Netti back cushions - with side supports. It can also be used separately.

Netti side support Stabile is mounted to the wheelchair chair-back profiles and is height and angle adjustable. It can be swung aside to make access to the user more convenient.



Item	Item number
Side support Stabile Left	26830
Side support Stabile Right	26843

MOUNTING

Netti side support Stabile is prepared for mounting to round and square backrest tubes. The black enclosed plastic bushing is used when mounting to square backrest tubes.

6 mm Allen key and a sharp knife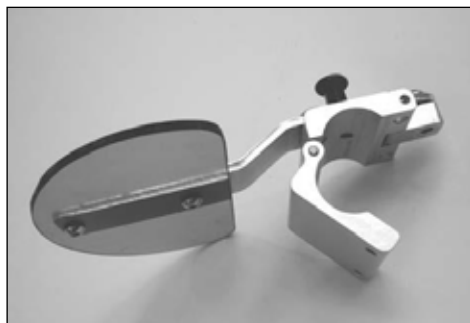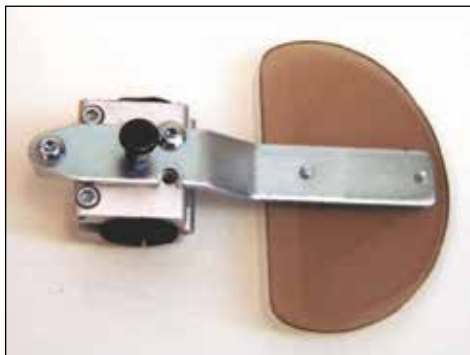

The Netti Side supports Stabile can be fitted with a cover.

- Open the Velcro on the cover
- Pull it over the side support plastic plate
- Fix it with the Velcro

After fitting the backrest cushion;

- Find the correct height for the side support. The support should be parallel with the integrated side support of the backrest cushion.
- If the Velcro back has a cut-out for the side support, place the side support there.
- If there is no opening in the Velcro back, cut a piece of the Velcro backrest to give room for the clamp to fit around the backrest tube at desired height.
- When mounting on a square tube, place the enclosed plastic bushing around the backrest tube before fixing the clamp around the tubing.
- Mount the clamp and tighten the 6 mm screws firmly.

Before the screws are tightened, you can choose the angle of the clamp and thereby increase or decrease the stability.

To release the side support and swing it outward, pull the release button out.

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Meyra GmbH