Netti[®] *Side support BASIC*

Mounting Description - uk

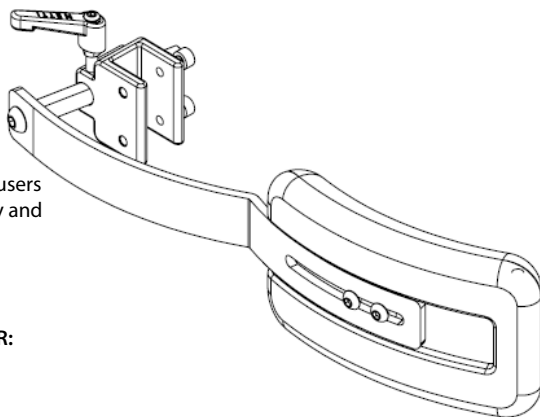
Montageanleitung - de

Monteringsanvisning - se

*inspire
joy of life*

NETTI SIDE SUPPORT BASIC (UK)

Netti side support Basic is especial designed for users in need for direct support of upper body stability and position.

1 MOUNTING INSTRUCTION FOR NETTI SIDE SUPPORT BASIC- FITTING FOR:

NETTI WHEELCHAIRS WITH SQUARE BACK SUPPORT TUBE

Item number 86145



 **Netti side support Basic can be adjusted in height, depth and sideways.**

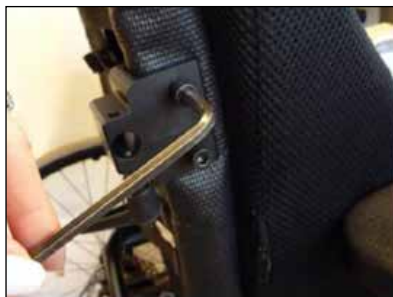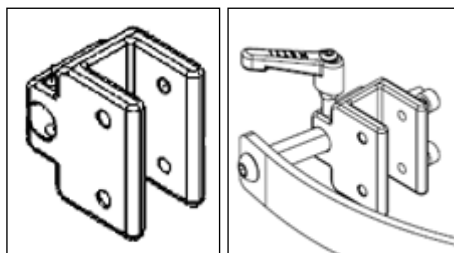Before you mount the side support to the wheelchair you need to define the long lasting seating position of the user.

- Find the desired position and mark the height at the back tube where you want to position the side support.
- Push the clamp over Velcro and back tube.
- It is recommended to position the clamp with the Italian grip on top as shown in the illustration on top of this page.
- The 2 screws fixing the clamp to the back tube should be inserted from the inside.
- Tighten the screws after mounting.

 **5 mm Allen key**

Insert the side support horizontal rod into the fixing bracket. Fix the side support cushion as far inward as desired. It is easily adjusted by loosening the Italian grip and slide the cushion in- or outward.

2 MOUNTING INSTRUCTION FOR NETTI SIDE SUPPORT BASIC - FITTING FOR:

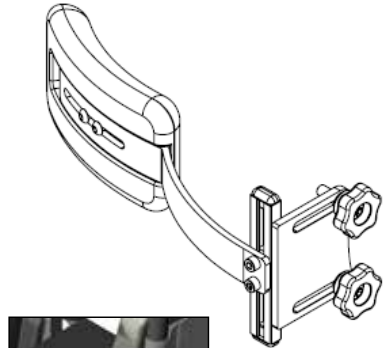
NETTI WHEELCHAIRS WITH ROUND BACK SUPPORT TUBE (NETTI 4U CE)

Item number 86146

Netti side support Basic can be adjusted in height, depth and side wards.



MOUNTING TO THE BACK

- At the back you will find 2 threaded inserts underneath the back fabric on each vertical back tube.
- To mount the star wheels to the back you need to make 2 holes in the fabric with a screwdriver where the inserts are.
- Mount the adapter plate with the 2 star wheels at the back in the holes you just made.
- With the adapter plate you can adjust the side support sideways.


HEIGHT ADJUSTMENT

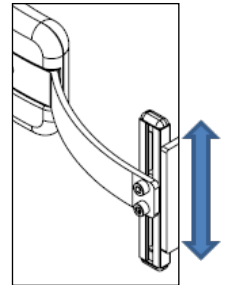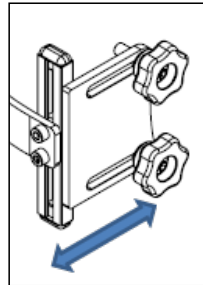- To adjust the side support height you need to use the vertical adapter.
- Before adjusting the height please define the long lasting seating position of the user.
- Loosen the 2 screws and adjust the desired height.
- Tighten the screws firmly.

DEPTH ADJUSTMENT:

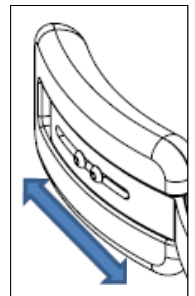- You can adjust the side support pad along the slotted hole on the outside of the side support.
- Loosen the 2 screws and adjust the desired depth.
- Tighten the screws firmly.

 **5 mm Allen key**

3 MOUNTING INSTRUCTION

FOR NETTI SIDE SUPPORT BASIC- FITTING FOR:

NETTI WHEELCHAIRS WITH ROUND BACK TUBE

Netti 4U BASE

Item number 88468

The side support Basic is height, width and depth adjustable.

Open the clamp

4 mm Allen key

Place the 2 clamp half's around the back tube of the wheelchair.

Keep all screws loose (to be tightened after fitting with side support)

Position the clamp in the area shown on the right. Adjust the clamp at the desired height and tighten the screws so that the clamp can be positioned later.

Turn the Italian grip to open the horizontal hole in the clamp.

You may also need to unscrew the grub screw on the bottom of the clamp by using

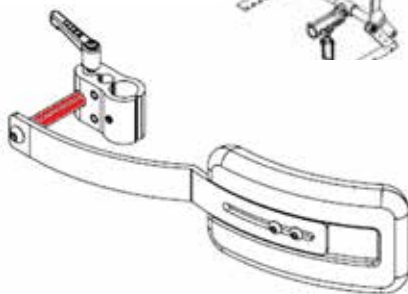
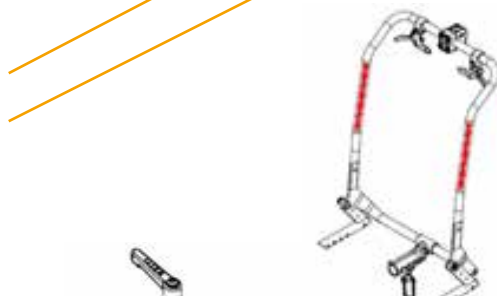
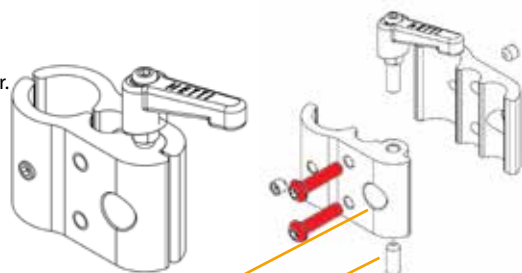
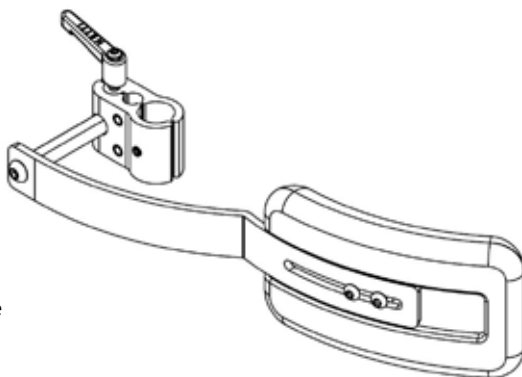
3mm Allen key

Screws on side support clamp kit 88295 should be tightened enough to avoid rotation of side support when this side support is fitted on the round back tube.


Insert the side support to the clamp and turn the lever clockwise to fix the side support in the clamp - see illustration on top of this page.

As soon as the user of the wheelchair sits in the wheelchair, position the side support in the desired height and width.


Adjust the angle of the side support on the backrest tube, by moving the side support horizontal rod inwards or outwards.

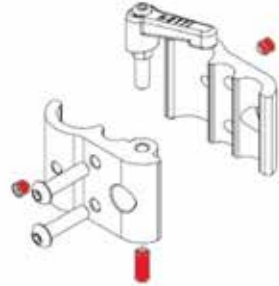As soon as you have finished the adjustment, tighten the 2 screws firmly.

i The position of the clamp can be secured with the 2 at the front of the clamp additional grub screws.

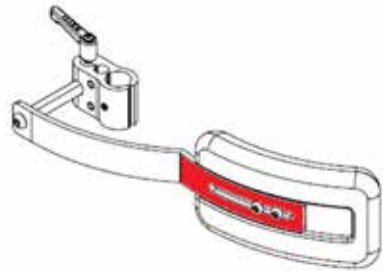i To prevent unauthorized removal of the side support, you can tighten the screw opposite of the clamp lever with a 3 mm Allen key.

SIDE SUPPORT BASIC DEPTH ADJUSTMENT:

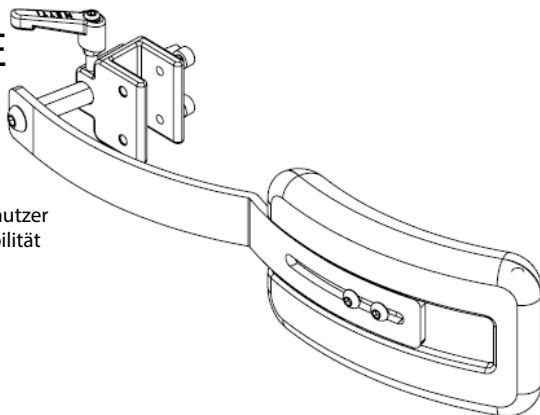
- Loosen the 2 screws at the back of the side support pad and move the pad to the desired depth.
- Tighten the 2 screws firmly.

i You can use the side support for Netti round tube clamp on the left or the right side.

To use the pad for the left or right side, turn the clamp 180° and mount the clamp lever on top and the grub screw at the bottom.

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Meyra GmbH