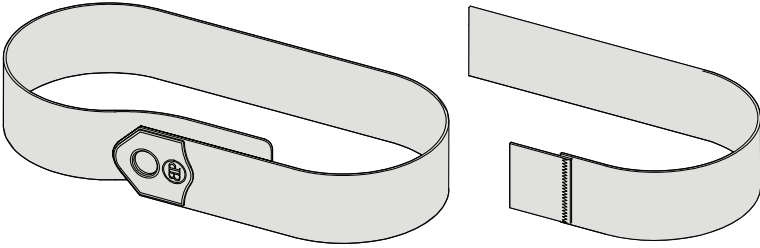
Elastic Chest Strap

INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. SP110S, SP110L, SP110XL, SP111S, SP111L & SP111XL

These instructions provide important information for the safe use and maintenance of all Bodypoint Elastic Chest Straps. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

⚠ WARNING! Product should be installed and fitted by a qualified rehab technician.

⚠ WARNING! Because of the risk of choking, it is dangerous to use this product without stabilizing the pelvis – always use with a properly fitted pelvic support belt.

USE:

⚠ WARNING! This product should only be used for positioning a person in a wheelchair or work chair. It is NOT intended for use as a transportation safety device, as a personal restraint device, or in any other application where its failure could result in injury. Misuse of this product is unauthorized and unsafe.

⚠ WARNING! Accidental release of this product can allow the user to slip down or fall from the wheelchair. If the user's movements or cognitive abilities could lead to accidental release, a caregiver must be present at all times during its use. Ensure that all caregivers know how to unfasten the product. Failure to do so may delay release in an emergency.

⚠ WARNING! As with any new seating support, this product may change the way a person sits. Users must continue to practice regular pressure relief activities and skin integrity checks, not only where this product contacts the user, but also in primary pres-

sure-bearing areas such as the sacrum, legs, and buttocks. If increased skin redness or irritation occurs, discontinue use and consult your physician or seating specialist. Failure to do so may result in serious injury, such as pressure ulcers.

MAINTENANCE: Check periodically for signs of wear in the stitching, webbing, and pads. If significant wear is found, contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons – health and safety depend on it!

🧺🔥📺🧼 CLEANING: Machine wash, hot, 60°C (140°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

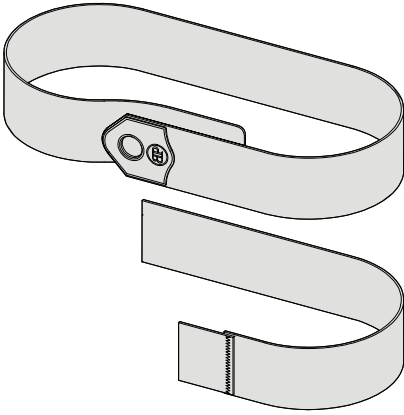
WARRANTY: This product carries a limited-life-time warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA go to www.bodypoint.com.

Bodypoint Europe BV
Kerkstraat 29
7396 PD Terwolde
The Netherlands



PACKAGE CONTENTS

Package may contain one or more of the following:

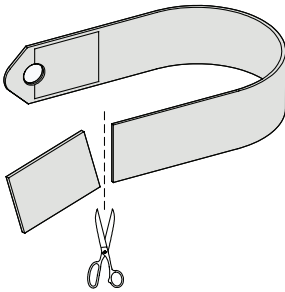
- Elastic Strap
- Elastic Extension Strap

To make shorter, cut to length.

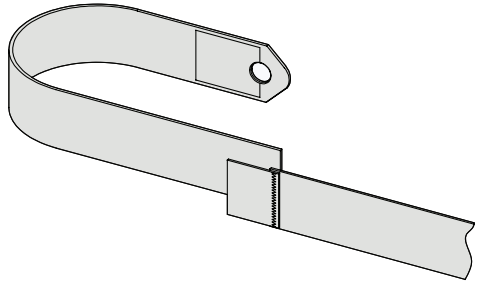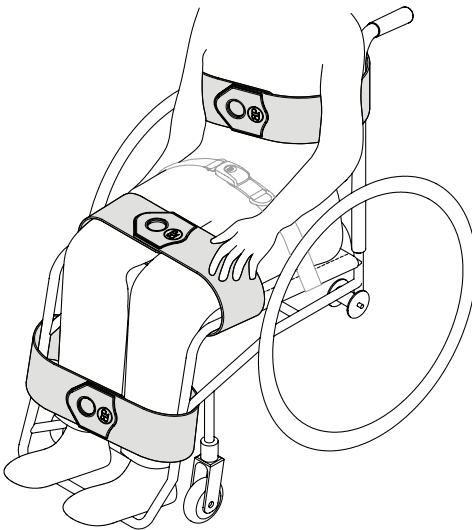To make longer, add elastic extension strap.

POSITIONING

Elastic straps provide temporary positioning support, and can be placed in multiple locations to suit the user's needs. For long-term positioning solutions consider more individualized products offered by Bodypoint.

SAFETY CHECK

When properly positioned in wheelchair, have user lean forward and side to side to check fit. Check for:

1. Comfort – If any rubbing occurs relocate straps.
2. Position - If too high or too low, adjust location.
3. Interference with other devices – Relocate straps as necessary to clear armrests, accessory pads, or feeding tubes.
4. Move chair through full range of motion, including tilt, folding, and rolling. Check for any interference.

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