Netti[®] *Side support Correction*

Mounting Description - uk

Montageanleitung - de

Monteringsanvisning - se

*inspire
joy of life*

NETTI SIDE SUPPORT CORRECTION (UK)

The side support Correction supports directly on the body and thereby offers extra stability together with the back cushion.

The side support Correction offer extra support at the side and at the front of the user. It has a comfortable soft upholstery. The side support can be adjusted 3D: up-down, backward-forward and in-out. The front-part can be moved inward to "embrace" the user and give support at the front. This can be useful with users who have a forward tendency. The front-part position is fixed by tightening the screw on the top of the front-part hinge.

-  Due to the thickness of the upholstery, care needs to be taken when placed too high in the axillary area. In this position there is a risk of limiting the blood flow to the arm and / or creating discomfort.

ITEMS

Kit fitting for square back tubes:

- 91455 Right Netti CE Plus, Netti CED, Netti III, Netti III HD and Netti III XHD
- 91456 Left Netti CE Plus, Netti CED, Netti III, Netti III HD and Netti III XHD

Kit fitting for round back tubes:

- 91457 Right Netti 4U CE
- 91458 Left Netti 4U CE


MOUNTING

1 MOUNTING TO ROUND BACK TUBE (NETTI 4U CE)


- Fix the adapter (2) to the back tube - 2 holes on each side are prepared in the back tubes.
- Use the 2 screws in the kit.
- Fix the side support bracket to the adapter.
- Follow the description on next page for mounting the side support clamp and the side support itself.

-  5 mm and 6 mm Allen key and eventually a sharp knife for cutting excess velcro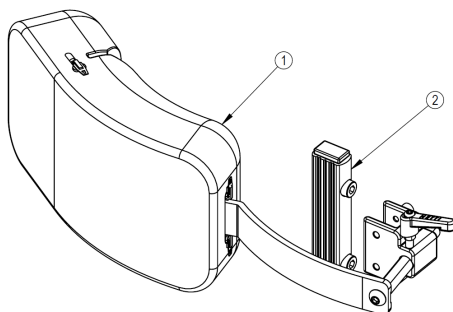

2 MOUNTING TO THE BACK TUBE

- Mount the side support fixing bracket in required height on the backrest tube.
- Fix both screws.
- For the Netti models loosen the straps on the velcro backrest fabric and twist the backrest material around the backrest tube to enable the adapter to slide on.
- Adjust the straps back to position. It might in some cases with very tight fabric be necessary to make a cut in the backrest fabric to fit the adapter. Alternatively, you can order a velcro with cut (Item 44992 or 44993)
- Slide the side support into the hole in the adapter and tighten the clamping screw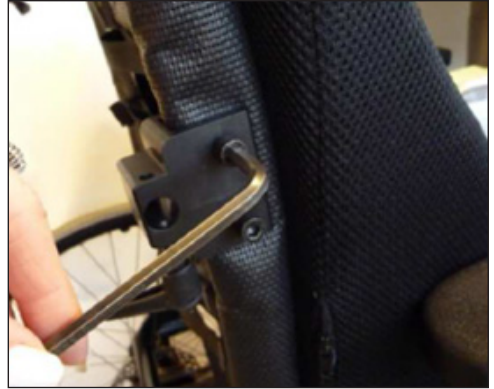


ADJUSTING THE SIDE SUPPORT LENGTH:

- Open the side support cover,
- Push the padding forward till you see the screws holding the side support arm
- Loosen the screws and slide / move the side support plate to desired position
- Tighten the screws, reposition the foam and close the cover.

 5 mm Allen key

ADJUSTING THE SIDE SUPPORT SIDEWAYS:

The depth is adjusted by loosening the Italian grip and pushing the horizontal side support rod into the fixing bracket.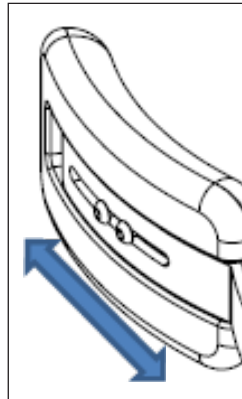

MOUNTING SIDE SUPPORT CORRECTION TO ADAPTPRO


kit 92514 - lower picture.

Insert 2 square nuts into the back profile of AdaptPro/Netti V back tube and fix the bracket in desired height.

 5 mm Allen key

REMOVAL OF THE SIDE SUPPORT FOR TRANSFER:

Loosen the Italian grip and pull the side support out of the mounting bracket.

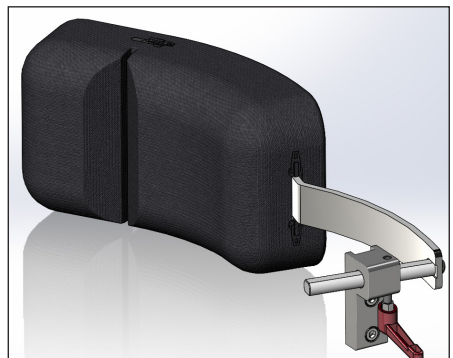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Meyra GmbH