

MOUNTING DESCRIPTION

MONTERINGSANVISNING / MONTAGEANLEITUNG /
DESCRIZIONE DELMONTAGGIO / NOTICE DE MONTAGE

HOFTEBELTE / HIP BELT / BECKENGURT /
CINTURA PELVICA / CINTURA VENTRALE

NETTI III, NETTI V, NETTI 4U CE, NETTI 4U CED



CE This product conforms to regulation MDR
(EU)2017/745 for medical products.

MD0052 ML 2025-06

*inspire
joy of life*

My-Netti.com

MONTERINGSANVISNING



2stk 13 mm fastnøkler

1. Trekk beltet gjennom hoftebeltefestet (dette er formontert på Netti Mini).
2. Tre beltet tilbake gjennom belteklemmen.
3. Fest hoftebeltet i det bakerste hullet i rygghengslebraketten med den vedlagte skrue og mutter.
4. Hoftebeltebraketten bør stå i 45° vinkel.

1



MOUNTING DESCRIPTION



2pcs 13 mm open-end spanners.

1. Pull the belt through the hole in the hip belt bracket .
2. Thread the belt back through the enclosed belt clamp.
3. Fix the hip belt bracket in the bacmost hole of the backrest hinge using the enclosed screws and nuts.
4. The hip belt bracket should be mounted in a 45° angle.

2



MONTAGEANLEITUNG



2 Stück 13 mm Maulschlüssel

1. Gurt durch die Öffnung des Gurthalters ziehen.
2. Gurt durch die Gurtschnalle zurückziehen.

3. Gurthalter mit Hilfe der beige-fügten Schraube und Muttern im hinteren Loch des Rückenlehnen -scharniers montieren.
4. Den Gurthalter im 45°-Winkel am Rahmen montieren.

DESCRIZIONE DELMONTAGGIO



Chiave semplice 2 pz. 13 mm

1. Tirare la cinghia attraverso il foro nella staffa della cintura pelvica.
2. Infilare il retro della cinghia nella fibbia della cinghia.
3. Fissare la staffa della cintura pelvica allo snodo dello schienale nel foro più indietro, utilizzando le viti e i dadi inclusi.
4. La staffa della cintura pelvica dovrebbe essere montata ad un angolo di 45°.

NOTICE DE MONTAGE



2 clés anglaises de 13 mm

1. Faites passer la ceinture dans la patte de fixation.
2. Faites repasser la ceinture dans l'attache.
3. Montez la patte de fixation dans le dernier trou à l'arrière de la ferrure du dossier, à l'aide des vis et écrous livrés.
4. Orientez la boucle à environ 45°.

3



4





This mounting description is available
to be downloaded from our homepage
www.My-Netti.com

FEEL FREE TO CONTACT US:

 Manufacturer of Netti:

Alu Rehab AS
Bedriftsvegen 23
N-4353 Klepp Stasjon
Norway

post.klepp@meyragroup.com
T: +47 51 78 62 20
my-netti.com

EU Distributor:

Alu Rehab ApS
Kløftehøj 8
DK-8680 Ry
Denmark

info.ry@meyragroup.com
T: +45 87 88 73 00
F: +45 87 88 73 19
my-netti.dk