



**EVOFLEX BELT
FASTENER KIT**

Mounting Descriptions



*inspire
joy of life*

EVOFLEX KIT

Item number 45021 xs,
82788 M, 82789 L

- Mounting description for fixing Evoflex belt to a dynamic seat plate on a Nett-Dynamic wheelchair.
- The belt kits consist of the belt - 2 pc




+ 2 nuts and 2 screws:



The dynamic seat plate comes with a rail under the seat plate - on each side.


A bracket for mounting the belt is attached to the rail on each side.

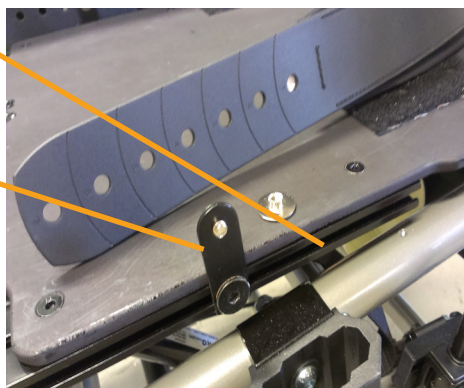

The screw from the Evoflex kit is pushed through the bracket and the belt - from the inside of the seat.

The belt is threaded unto the screw - using the belt hole that give the correct seatbelt length for the user.

Please pay attention to use the same hole on both half's of the belt to fit the buckle in the centre.

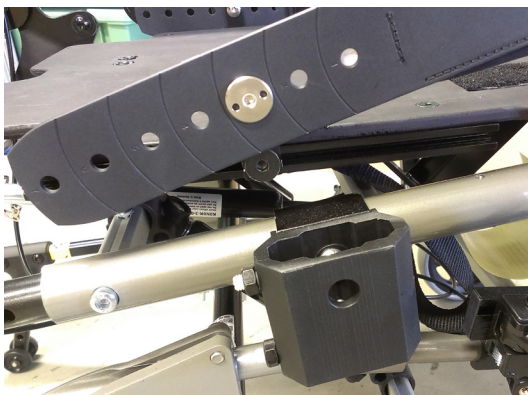

The nut is pushed through the belt onto the screw - from the outside of the chair.

Tighten the screw and nut well.

Please note that the nut has a length that allows for free movement of the belt when screw and nuts are tightened.



With the belt fixed to the rail on the seat plate the seat plate can swing up and follow the user when he/she is stretching.

Adjust the position of the belt by loosening the belt bracket screws and sliding the bracket to required position on the rail.

The belt position must be forward that it crosses the thighs of the user without squeezing vital parts.



Manufacturer of Netti:



Alu Rehab AS
Bedriftsvegen 23
N-4353 Klepp Stasjon
Norway



post.klepp@meyragroup.com
T: +47 51 78 62 20
my-netti.com

EU Distributor

Alu Rehab ApS
Kløftehøj 8
DK-8680 Ry
Denmark

info.ry@meyragroup.com
T: +45 87 88 73 00
F: +45 87 88 73 19
my-netti.dk