



Netti[®] *Head support fixations*

Mounting Descriptions



*inspire
joy of life*

CONTENTS

1.	INTRODUCTION	3
2.	NETTI UNIVERSAL HEAD SUPPORT ADAPTER	4
2.1	ALU HEAD SUPPORT ADAPTER	5
2.2	ADAPTPRO HEADSUPPORT ADAPTER	5
2.3	NETTI V HEADSUPPORT ADAPTER	6
3.	MEYRA MEYLIFE HEAD SUPPORT	7
3.1	NETTI III AND 4U WHEELCHAIRS.	7
3.2	MEYRA MEYLIFE HEAD SUPPORT ON NETTI V	7

1. INTRODUCTION

Netti wheelchairs can be equipped with four different head support fixation brackets:

Netti universal and Alu head support adapter

Adapt Pro head support adapter is only compatible with Netti Adapt Pro wheelchair, which again can be combined with all Netti head supports.

MyLife head support with Meyra adapter fits to all Netti wheelchairs, except Adapt Pro

Important points in this manual are marked with the following symbols:

 **Symbol of forbidden actions.**
No warranty can be claimed whenever these actions are implemented.

 **Symbol of warning.**
Whenever this symbol is used, caution has to be taken.

 **Symbol for important information.**

 **Symbol for useful tips.**

 **Symbol for tools.**

 **Please study this mounting description and pay careful attention to the user manual for the wheelchair where the head supports is mounted.**

 **User and attendant need instruction and training to mount and adjust the head supports.**

 **ADJUST THE WHEELCHAIR TO THE USER:**

Adjust seat depth and eventually the wheelchair balance, foot board height, armrest height, head support height and depth, chair back cushion height.

 **ANTI-TIP**
Correctly fitted, the anti-tip will secure the chair from tipping backwards. Please check the position of the anti-tips and adjust if the chair is unstable when tilted.

For more information about adapting the wheelchair to the user please see www.My-Netti.com knowledge and tools.

 **Illustrations may differ from the delivered product.**

 **For visually impaired people, manuals and catalogues can be downloaded at www.My-Netti.com**

2. NETTI UNIVERSAL HEAD SUPPORT ADAPTER

All Netti head supports have the same functions .

1. Mounting the head support

- Mount the vertical bar in the head rest adapter.

2. Adjust the angle of the head support

- The angle of the head support can be adjusted with the star wheel at the back of the horizontal bar.
- Loosen the star-wheel, adjust the head support, and tighten the star wheel again.

3. Adjust the head support length.




- Loosen the clamping lever on the head support adapter, adjust the height and tighten the clamping lever again.

4. Head support depth adjustment.

- Loosen the clamping lever on the horizontal bar.
- Adjust to correct length, and tighten the clamping lever again.

5. Removal of the head support.

- Remove the head support by loosening the clamping lever on the head support adapter, and pull the vertical bar up and out of the head support adapter

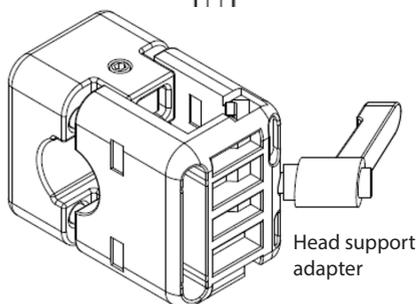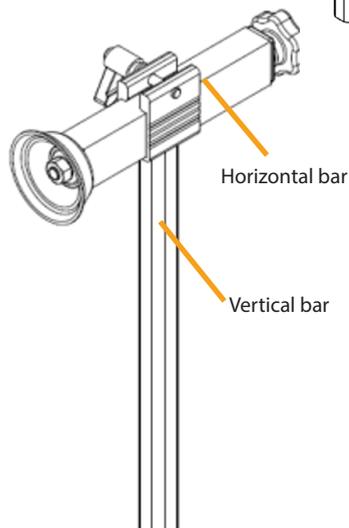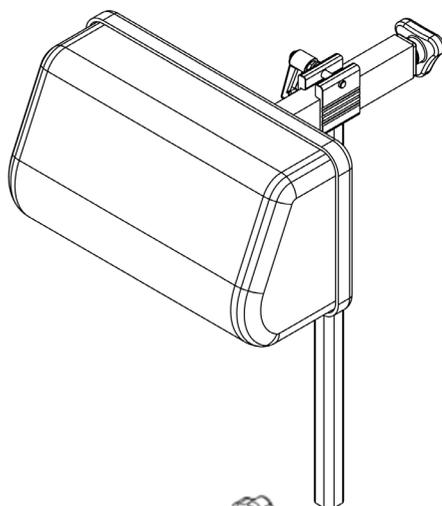


2.1 ALU HEAD SUPPORT ADAPTER

The Netti Alu head rest adapter is recommended to be mounted when the universal does not hold the forces of the user.

1. Mounting the head support


- Mount the vertical bar in the head rest adapter.

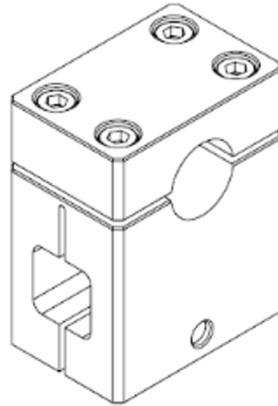

2.2 ADAPTPRO HEAD SUPPORT ADAPTER

The AdaptPro head support adapter (92901) is specifically designed to fit the Netti AdaptPro wheelchair. It is used for mounting various types of Netti head supports, including models A, B, C, D, E, F, as well as the Savant head support.

One of the key features of the AdaptPro head support adapter is that it offers the same adjustments as the Netti Universal head support adapter. This ensures that users can achieve a comfortable and secure fit, tailored to their individual needs.

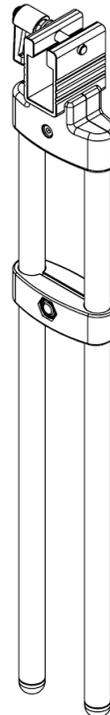


2.3 NETTI V HEAD SUPPORT ADAPTER

The Netti V head rest adapter is only used for the Netti V family wheelchairs.

This head support adapter can easily be adjusted sideways on the crossbar at the top of the back support

1. **Mounting the head support adapter.**
 - Mount the head support adapter to the push bar by clamping it with the two M6 screws.
2. **Mounting the head support**
 - Mount the vertical bar in the head support adapter and tighten the star-wheel..

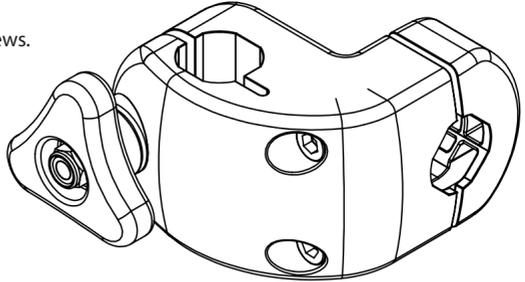


3. MEYRA MEYLIFE HEAD SUPPORT

3.1 MEYRA MEYLIFE HEAD SUPPORT ON NETTI III AND 4U.

The Meyra MEYLIFE head support is mounted on the Ø22mm push bar on Netti III and 4U wheelchairs.


1. Position the bracket on the Ø22mm push bar.
2. Insert 4x M6x30 screws into the bracket and tighten them securely.

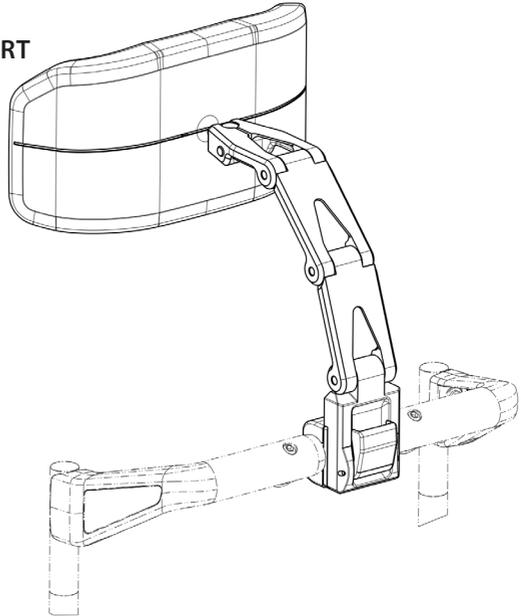

3.2 MEYRA MEYLIFE HEAD SUPPORT ON NETTI V

When mounting the Meyra MEYLIFE head support on Netti V you will need 95216 adapter.

1. Mount the Meyra MEYLIFE head support to the 95216 adapter. Use 4x M6x20 screws to secure the head support to the adapter.
2. Clamp the adapter on the upper crossbar of the Netti V back:
Attach the adapter to the upper crossbar in the same manner as you would with the standard head support adapter for the Netti V.

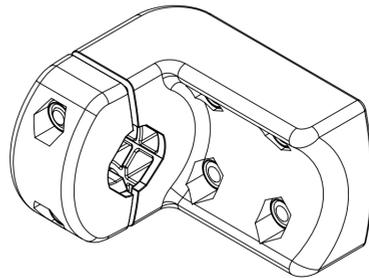

Manufacturer of Netti:

 Alu Rehab AS
Bedriftsvegen 23
N-4353 Klepp Stasjon
Norway

 post.klepp@meyragroup.com
T: +47 51 78 62 20
my-netti.com

EU Distributor

Alu Rehab ApS
Kløftehøj 8
DK-8680 Ry
Denmark

info.ry@meyragroup.com
T: +45 87 88 73 00
F: +45 87 88 73 19
my-netti.dk