Netti[®] **UNO SEAT, SIT & CONTOUR**

English	Instruction for use
Français	Manuel d'utilisation
Deutsch	Gebrauchsanweisung
Italiano	Istruzioni per l'uso
Norsk	Bruksanvisning
Dansk	Brugsanvisning
Svenska	Bruksanvisning
Polski	Instrukcja używania

CE This product conform to regulation MDR
(EU)2017/745 for medical products.

CONTENTS

1. About this manual
2. Intended Purpose
3. Compatibility / Application
4. Technical specifications
5. Warnings and Precautions
6. Installation Instructions
7. Maintenance
8. Inspection Procedures
9. Washing Instructions / Recycling
10. Testing & Warranty

 **Netti** Uno seat **Netti** Sit **Netti** Contour

1. About this manual

To avoid damage to the product and reduce the risk of improper use, read these Instructions for Use carefully before using the cushion.

 European Conformity

 Medical Device

 Date of manufacturing

 Item number

 Core

 Cover

 Max load seat cushion

 Machine wash at 65°C

 Hand wash at max 40 °C

 Do not spin

 Do not tumble dry

 Do not dry clean

 Do not iron

  Unique device identifier = GS1 Data Matrix

 Manufacturer - Name and address

 Read instruction for use

 Keep dry

 Keep away from sunlight

 Country of origin

 Ignition resistance

 Symbol of warning. Whenever this symbol is used, caution has to be taken.

 Symbol for important information.

 Intended for use in wheelchairs when used as seats in motor vehicles.

Please note that this manual is updated according to the year and date stated on each page. The latest version of these instruction for use is available at my-netti.com

For enhanced readability (advantageous for users with visibility challenges) please find our instruction for use on our web page: my-netti.com

Latest instruction for use updates, product safety notes, addresses and other product information like recalls etc. will be published on our web page.

2. Intended Purpose

Netti Uno Seat, Netti Sit and Netti Contour are non-powered wheelchair seat cushions intended to support seated positioning, pressure distribution and seating comfort for wheelchair users. The devices are intended to be used as seat cushions in compatible wheelchairs as specified by the manufacturer or assessed as suitable by a qualified healthcare professional.

Indications for use

The cushions are intended for users who require seating support, pressure distribution and/or improved sitting stability during wheelchair use.

Contraindications

Do not use the cushion if the user has a pressure injury, skin lesion or other tissue condition that prevents sitting, unless use has been specifically assessed and approved by an appropriate healthcare professional.

Patient target group

Children, adolescents and adults who use a wheelchair and require a seat cushion for pressure distribution, positioning and / or stability.

Intended users

The device is intended to be used by wheelchair users, caregivers and healthcare professionals. Cushion selection and first-time clinical assessment should be performed by a qualified healthcare professional when the user has pressure injury risk, impaired sensation, postural instability, pain, or a history of skin breakdown.

Intended use environment

The device is intended for indoor and outdoor use as part of wheelchair seating.

Expected clinical/user benefits

Depending on model and correct use, expected benefits may include:

- improved pressure distribution,
- improved seating stability,
- improved positioning support,
- improved seating comfort,
- support for a favorable seating microclimate.

3. Compatibility / Application

The cushion shall only be used in a wheelchair seat that provides suitable dimensions and stable support for the cushion. Do not assume compatibility with every wheelchair model. If the cushion is used with a wheelchair not supplied or validated by Alu Rehab, compatibility and safe use shall be assessed by a qualified healthcare professional or otherwise documented by the responsible economic operator.

4. Technical Specifications

Netti Uno Seat

Core

4 layers of polyurethane foam. The cushion is crafted from polyurethane foam with varied weight-bearing properties.

Two layers are placed side-by-side horizontally in the middle section. The rear features a softer foam layer, while the front is reinforced with a firmer layer, helping to stabilize the pelvis at the front of the seating bones.

Netti Sit

Core

Four layers of polyurethane foam. The cushion is designed with polyurethane foam layers that each have different support properties. In the middle layer two of these layers are placed side by side horizontally. The back of the cushion features a softer foam, while the front uses a firmer layer. This helps stabilize the pelvis at the front edge of the sitting bones.

On top of the bottom support foam, there's a textile layer that binds the foam's surface together. This helps reduce the cushion's weight while boosting its load-bearing ability.

Side supports are integrated into the outer edge of the core, increasing lateral support and maximizing the contact area.

Netti Contour

Core

The cushion is made from one layer anatomically shaped polyurethane which enables different weight carrying and side stabilizing properties and gives integrated abduction and lower back edge.

Outer cover: Netti Uno Seat, Sit & Contour

Option 1: 3D Option

Crafted from 100% knitted polyester with a 3D texture. The cover is extremely durable, stretchy, and provides excellent airflow. Only Netti Sit with 3D outer cover has an inner cover named Netti Protex. Here the core is wrapped in a white protective cover, Protex, that keeps it free from dirt. Protex is a semi-permeable polyurethane material that allows it to "breathe".

This means water vapor (water molecules) can pass through, while liquid droplets are blocked from entering. Protex is also highly flexible.

Option 2: Easycare (EC)

100% polyester, highly stretchable, easy to clean or wipe down, and provides good breathability.

A friction fabric with sewn-on Velcro is attached to the underside of the outer cover, ensuring the cushion stays securely in place in the wheelchair, regardless of wheelchair model or surface.

The therapist is responsible for selecting the appropriate cushion for the wheelchair user.

5. Warnings and Precautions

- Use the cushion only in the correct orientation, as indicated on the label. Incorrect positioning may reduce pressure-distribution performance and user stability.
- Do not use the cushion if the cover is damaged, if the foam core is torn, permanently deformed, contaminated, or shows signs of excessive wear. Replace damaged parts before further use.
- Users with existing skin damage, reduced sensation, impaired mobility, postural instability, or high risk of pressure injury should use the cushion only after assessment by an appropriate healthcare professional.
- Check the skin regularly, especially during the first days of use and after any change in sitting time, body weight, posture, wheelchair setup or cushion condition.
- If redness persists for more than 30 minutes after unloading, discontinue use and contact a healthcare professional.
- Regular pressure relief remains necessary. This cushion does not eliminate the need for repositioning or pressure-relief routines.
- Do not modify the cushion or use non-approved replacement parts or non-approved covers, as this may affect safety and performance.
- If the cushion is used in a wheelchair that serves as a seat in a motor vehicle, use is permitted only where the relevant wheelchair and cushion combination has been validated for that purpose.

6. Installation Instructions

Remove the cushion from its packaging and place it on the wheelchair seat.

Position the cushion according to the label sewn onto the underside of the cover.

Make sure the friction fabric with the label faces down toward the seat plate. Attach the opposite Velcro to the seat plate.

Ensure the Velcro on both the seat plate and the bottom of the foam cushion connect securely for a safe fit.

The front of the seat cushion should be in line with the front of the seat plate.

7. Maintenance

The only maintenance required for the cushions is washing:

1. Unzip the back of the cushion and remove the cover.
2. If you have a Netti Sit with 3D cover, unzip the underside of the middle cover.
3. Take out the core and stand it upright to air out while the cover is being washed.
4. Follow the washing instructions found on the label sewn to the underside of both the middle and outer covers.
5. When replacing the core, make sure the label faces downward. For correct front/back placement, refer to the label.
6. Attach the middle cover, make sure the zipper is at the back and the label faces down.
7. Attach the outer cover, ensure the zipper is at the back and the label faces down.

8. Inspection Procedures

On a regular basis, check that the core and covers are placed correctly.

Before sitting for long periods, begin by using your cushion for short sessions. If you have any pressure injury (PI) concerns, start 5 minutes for the first use. After each session, check the skin areas that were in contact with the cushion. If you notice any redness, it should disappear within 10 to 30 minutes. If the redness lasts longer than 30 minutes, contact your doctor or therapist.

If your skin checks show no lasting redness, you can slowly increase your sitting time by 5-30 minutes to 1 hour each day, depending on your PI issues. Keep checking your skin after each increase. Generally, sitting for 2 to 4 hours at a time is the recommended maximum. Even after your skin gets used to the cushion, continue checking it at least once a day.

Make sure to regularly relieve pressure by lifting yourself or using a power tilt, as advised by your healthcare provider.

Ongoing checks and adjustments

Over time, your cushion may compress or your weight may change. Regular checks are necessary to ensure there's enough space under your sitting bones (ischial tuberosities). Not doing so could lead to injury.

Important warning

If redness from using the cushion lasts longer than 30 minutes, stop using the cushion and contact your doctor right away.

Follow-up care

It's recommended to have your posture and skin condition reviewed by a healthcare professional at least once a year, or sooner if you notice changes in your weight, posture, or overall health that might affect your sitting comfort and safety.

The need to wash the core and covers will vary from user to user depending on the degree of incontinence and other individual factors.

As a rule, remove the core from the covers once a week and let it stand upright to air out while the covers are being washed.

It is recommended to order a spare cover so that the seat cushion can remain in use while the original cover is being washed.

9. Washing Instructions / Recycling

	Netti Uno Seat / Netti Contour	Netti Sit
CORE		
Wash	Hand wash at 40 °C	
Desinfection, regular	Virkon S*. Remove visible dirt, spray the dirty part of the surface, wait 1-2 minutes and wipe.	
Drying	Tumble dry, air dry upright	
Inner Cover		
	Not standard	
Wash		Machine wash at 70 °C
Drying		Tumble dry up to 80 °C
Outer Cover		
Wash	Machine wash at 70 °C	
Desinfection, regular	Virkon S*. Remove visible dirt, spray the dirty part of the surface, wait 1-2 minutes and wipe.	
Drying	Tumble dry up to 80 °C	

Alu Rehab considers seat cushions to be a personal and intimate aid. For recirculation, it is recommended to at least replace the outer cover and, if needed, the inner cover of the cushion.

10. Testing & Warranty

Testing

Netti seat cushions are tested and have been approved for both indoors and outdoors use. Netti seat cushions are CE marked and are manufactured to comply with the Medical Device Regulation (EU 2017/745).

Maximum user weight for Netti Uno Seat, Sit & Contour

Seat cushion with seat width < 500 mm = 160 kg
 Seat cushion with seat width over 500 mm = 200 kg.

Netti wheelchairs with Netti cushions have been tested by an accredited German test laboratory according to EN 12183.

Netti wheelchairs with Netti cushions have also been crash tested at TASS International AS, Netherlands, and evaluated by accredited German test laboratory according to ISO 7176-19. Both are approved for being used as a seat in a car.

Netti Uno Seat, Sit and Contour seat cushions (covers) have been tested according to the following standards:
 Fire resistance: EN 1021-1 and 2.
 Skin contact: Cytotoxicity according to ISO 10993-5.
 The PU and PUR foams do not contain added bisphenols or phthalates.

Warranty

The durability and longevity of the Netti Uno Seat, Sit & Contour seat cushions depend on the design, selected hardnesses and stress applied. Nettis foam cushions (PU and PUR) are of very high quality. Therefore, there is a 2-year warranty on all Netti-labelled foam seat cushions.

 Alu Rehab is not responsible for any damage resulting from inappropriate or incorrect installation and / or repairs, wear, maintenance and washing as well as deliberate damage not approved by Alu Rehab or produced by third parties. In such cases, this warranty shall be considered null and void.

 This warranty is only valid when the user uses, maintains and handles the seat cushion as described in this instruction for use.

 The warranty is only valid when the user has followed our recommendations as described in "Routines for inspection".

Warranty claim

If a product develops a fault during the warranty period as a result of a design or manufacturing defect, you may submit a warranty claim.

- Claims shall be submitted as soon as the defect is discovered and no later than 2 weeks after discovery.
- Claims are to be addressed to the sales agent of the wheelchair. Please note that sales documentation has to be filled in and signed correctly with serial number, invoice number and eventual NeC number to document time and place of the purchase of the wheelchair.
- The sales agent and Alu Rehab are to decide whether a defect is covered by this warranty. The claimant will be informed of the decision as soon as possible.
- If the claim is accepted, the sales agent and Alu Rehab representative are to decide if the product will be repaired, replaced or if the customer is entitled to a reduced price.
- If a warranty claim is judged to be invalid - after

careful inspection of the defect (defect due to wrong use and/or lack of required maintenance) you are free to decide if you want to have the product repaired (if possible) at your expense, or if you want to purchase a new product.

 Normal wear, incorrect use or incorrect handling is not a reason for claims.

Combining with other products

Use of the cushion with wheelchairs or accessories not manufactured or supplied by Alu Rehab is permitted only where compatibility and safe performance have been verified.

Transport in motor vehicles

Statements regarding crash testing and use as a seat in a motor vehicle apply only to validated wheelchair and cushion combinations and under the conditions specified by the manufacturer.

PRODUCT LIABILITY

The Netti Uno Seat, Sit & Contour cushions are tested and risk-assessed by Alu Rehab.

Substitutions or alterations by third part suppliers to Netti seat cushions require risk evaluation and acceptance of the product responsibility and safety for use of the wheelchair from the manufacturer that is performing the substitution or alteration.

 Information on productsafety notices and product recalls are available at my-netti.com

 Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Environmental care and waste disposal

Alu Rehab and its suppliers wish to protect the environment.

This means:

- That we avoid using environmentally harmful substances and processes to the greatest extent possible.
- That Alu Rehas products ensure a long service life and a high degree of flexibility - to benefit the environment and economy.
- That all packaging can be recycled.
- That seat cushion was designed to make recycling easier.
- Please contact your local recycling firm to get updated information about which components and materials can be recycled and how they are separated.

A recycling manual for Netti wheelchairs can be downloaded at my-netti.com

For visually impaired people, manuals and catalogues can be downloaded at my-netti.com

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