Netti[®] *Precision seat cushions*

English	Instructions for Use
Français	Instructions d'utilisation
Deutsch	Gebrauchsanweisung
Italiano	Istruzioni per l'uso
Norsk	Bruksanvisning
Dansk	Brugsanvisning
Svenska	Bruksanvisning
Polski	Instrukcja używania

CE This product conform to regulation MDR
(EU)2017/745 for medical products.

*inspire
joy of life*

CONTENTS

1. ABOUT THIS MANUAL
2. INTENDED PURPOSE
3. COMPATIBILITY / APPLICATION
4. TECHNICAL SPECIFICATIONS
5. WARNINGS AND PRECAUTIONS
6. INSTALLATION INSTRUCTIONS
7. INSPECTION PROCEDURES
8. WASHING INSTRUCTIONS / RECYCLING
 - 8.1 Cleaning and washing
 - 8.2 Reuse
9. TESTS & WARRANTY
 - 9.1 Tests
 - 9.2 Warranty
 - 9.3 Claim
 - 9.4 Combination with other products
 - 9.5 The environment and waste disposal

1. About this manual

In order to avoid damages while using the Netti Precision seat cushion, please read this instructions for use carefully before starting to use the cushion.

 European Conformity

 Medical Device

 Date of manufacturing

 Item number

 Core

 Cover

 160 kg Max load seat cushion

 Machine wash at 65°C

 Hand wash at max 40 °C

 Do not spin

 Do not tumble dry

 Do not iron

 Unique device identifier = GS1 Data Matrix

 Manufacturer - Name and address

 Consult instruction for use

 Keep dry

 Keep away from sunlight

 Date of manufacturing

 Ignition resistance

 Symbol of forbidden actions. No warranty can be claimed whenever these actions are implemented

 Symbol of warning. Whenever this symbol is used, caution has to be taken

 Symbol for important information

 Symbol for tools

 Intended for use in wheelchairs when used as seats in motor vehicles.

Please note that this manual is updated according to the year and date stated on each page. The latest version of these Instruction for Use is available at my-netti.com

For enhanced readability (advantageous for users with visibility challenges) please find our instructions for use on our web page: my-netti.com
Latest instructions for use updates, product safety notes, addresses and other product information like recalls etc. will be published on our web page.

2. Intended Purpose

Netti Precision are non-powered wheelchair seat cushions intended to support seated positioning, pressure distribution and seating comfort for wheelchair users. The devices are intended to be used as seat cushions in compatible wheelchairs as specified by the manufacturer or assessed as suitable by a qualified healthcare professional.

Indications for use

The cushions are intended for users who require seating support, pressure distribution and/or improved sitting stability during wheelchair use.

Contraindications

Do not use the cushion if the user has a pressure injury, skin lesion or other tissue condition that prevents sitting, unless use has been specifically assessed and approved by an appropriate healthcare professional.

Patient target group

Children, adolescents and adults who use a wheelchair and require a seat cushion for pressure distribution, positioning and/ or stability.

Intended users

The device is intended to be used by wheelchair users, caregivers and healthcare professionals. Cushion selection and first-time clinical assessment should be performed by a qualified healthcare professional when the user has pressure injury risk, impaired sensation, postural instability, pain, or a history of skin breakdown.

Intended use environment.

The device is intended for indoor and outdoor use as part of wheelchair seating.

Expected clinical/user benefits.

Depending on model and correct use, expected benefits may include:

- improved pressure distribution;
- improved seating stability
- improved positioning support,
- improved seating comfort,
- support for a favorable seating microclimate.

3. Compatibility / Application

The Netti Precision seat cushions shall only be used in a wheelchair seat that provides suitable dimensions and stable support for the cushion. Do not assume compatibility with every wheelchair model. If the cushion is used with a wheelchair not supplied or validated by Alu Rehab, compatibility and safe use shall be assessed by a qualified healthcare professional or otherwise documented by the responsible economic operator.

Model:

NETTI PRECISION 1

- When user is at risk of skin and pressure injuries.
- Pressure distribution and relief.
- Protects bony areas.
- Support of the pelvic ensures a good seating position.
- Great immersion minimises risk of sliding forward.

NETTI PRECISION 2

- For users with sensitive skin, high risk of skin damage or even skin injuries.
- Pressure distribution and relief.
- Enhanced pressure relief of tail bone and pelvic floor.
- Support of pelvic ensures a good seating position.
- Great immersion minimises risk of sliding forward.

NETTI PRECISION ACTIVE


- Great microclimate control
- Unique airflow
- Pressure distribution and relief across seating surface
- Protects bony areas including sit bones and tailbone
- Pelvic support ensures good seating position and stability
- Shock absorption reduces risk of pressure injuries
- Hygienic & quick dry core

CREATED WITH ERGOMISE™ TECHNOLOGY

With Ergomise™ Technology we ensure that our cushions deliver proven results. The Netti Precision™ seat cushions and the underlying technology have been tested against competitor cushions not only by Netti, but also by the test institute at the University of Pittsburgh (ISO 16840-2, -6,-12-13), and the results show outstanding performance.


TEMPERATURE CONTROL

High temperature can lead to humidity causing skin breakdown. The Netti Precision™ material maintain a low temperature and thus lower risk of injury. According to the test institute at the University of Pittsburgh, the temperature at the interface between the user and Netti Precision™ material is only 32,7°C after 3 hours. Comparable cushions measure between 33,7°C and 35,7°C.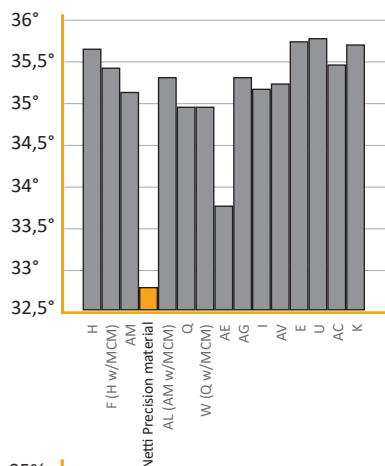

HUMIDITY CONTROL


Humidity above 60% can increase skin vulnerability and cause friction between the skin and textiles, which potentially can lead to pressure injury. This is why we aim to keep the humidity low. The tests from the University of Pittsburgh show that the cushion with Netti Precision™ material has a humidity level of 62% after 3 hours compared to an average of 81,2% for the 14 comparable cushions.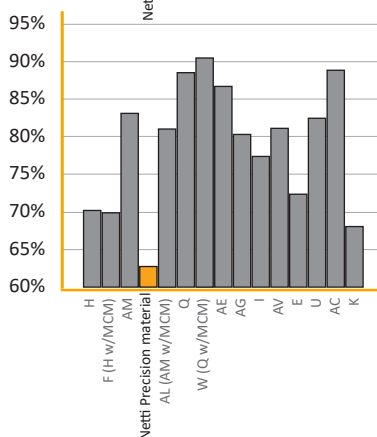

PEAK PRESSURE CONTROL

Once a decubitus ulcer has developed, wheelchair users can face painful treatments, months of bed rest, and hospitalisation. Netti Precision™ offers an effective pressure redistribution strategy, which is key to prevention of pressure ulcers.

Pressure mapping has shown that Netti Precision™ surpasses other pressure-relieving cushions when it comes to peak pressure reduction.

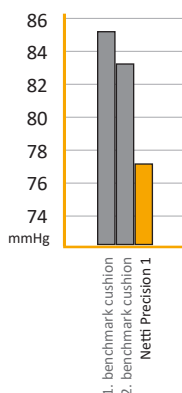4. Technical specifications

CORE

The core is made of bio-compatible thermoplastic polyurethane (TPU). The material has no added phthalates.

OUTER COVER

You can choose between 3 different fabrics:

- Breathable and stretchable Stella+ with a foam insert
- Wipeable, breathable EasyCare+ with a foam insert.
- Incontinence, open structure with Stella at the top, 100% water sealed at the bottom, no leaking on the wheelchair

TOTAL WEIGHT:

Seat width 350 mm - 0.7 kg

Seat width 600 mm - 1.9 kg

 The therapist is responsible for choice of cushion model for the wheelchair user. If the standard solutions does not cover your needs, please contact our customer service for Netti customized solutions.

TEMPERATURE RANGE:

Precision cushions can be used safely in temperature range between -10 °C and up.

The cushion should not be used under -10 °C because it will harden the material. We recommend keeping the seat cushion under 65 °C at all times.

5. Warnings and Precautions

- Use the cushion only in the correct orientation, as indicated on the label. Incorrect positioning may reduce pressure-distribution performance and user stability.
- Do not use the cushion if the cover is damaged, if the foam core is torn, permanently deformed, contaminated, or shows signs of excessive wear. Replace damaged parts before further use.
- Users with existing skin damage, reduced sensation, impaired mobility, postural instability, or high risk of pressure injury should use the cushion only after assessment by an appropriate healthcare professional.
- Check the skin regularly, especially during the first days of use and after any change in sitting time, body weight, posture, wheelchair setup or cushion condition.
- If redness persists for more than 30 minutes after unloading, discontinue use and contact a healthcare professional.
- Regular pressure relief remains necessary. This cushion does not eliminate the need for repositioning or pressure-relief routines.
- Do not modify the cushion or use non-approved replacement parts or non-approved covers, as this may affect safety and performance.
- If the cushion is used in a wheelchair that serves as a seat in a motor vehicle, use is permitted only where the relevant wheelchair and cushion combination has been validated for that purpose.

6. Installation instructions

The seat cushion is taken out of the packing and placed in the seat of the wheelchair. Place the cushion according to instructions on the labels attached to the bottom fabric of the outer cover. Check that the Velcro and friction fabric is facing down towards the seat plate.

The seat plate needs to have the opposite Velcro firmly fixed. Make sure that the Velcro from the seat plate and the bottom of the Precision cushion must fit direct on each other to ensure safe positioning of the cushion. The front of the cushion should be lined up even with the front of the seat plate.

Small adjustments may be made to fit the cushion to non-Netti wheelchairs. For back tubes, cut-outs up to 40 x 40 mm may be made in the TPU core corners using a sharp, long-bladed knife. Modifications must

be minimal and not affect cushion function.

7. Inspection Procedures

 On a regular basis, check that the core and covers are placed correctly.

Before sitting for long periods, begin by using your cushion for short sessions. If you have any pressure injury (PI) concerns, start 5 minutes for the first use. After each session, check the skin areas that were in contact with the cushion.

If you notice any redness, it should disappear within 10 to 30 minutes. If the redness lasts longer than 30 minutes, contact your doctor or therapist.

If your skin checks show no lasting redness, you can slowly increase your sitting time by 5-30 minutes to 1 hour each day, depending on your PI issues. Keep checking your skin after each increase. Generally, sitting for 2 to 4 hours at a time is the recommended maximum. Even after your skin gets used to the cushion, continue checking it at least once a day.

Make sure to regularly relieve pressure by lifting yourself or using a power tilt, as advised by your healthcare provider.

 **Important Warning:**

If any redness from using the Precision cushion stays longer than 30 minutes, stop using the cushion and contact your healthcare provider immediately.

 **Ongoing Checks and Adjustments:**

Over time, your cushion may compress or your weight may change. Regular checks are necessary to ensure there's enough space under your sitting bones (ischial tuberosities). Not doing so could lead to injury.

 **Follow-Up Care**

It's recommended to have your posture and skin condition reviewed by a healthcare professional at least once a year, or sooner if you notice changes in your weight, posture, or overall health that might affect your sitting comfort and safety.

The need to wash the core and covers will vary from user to user depending on the degree of incontinence and other individual factors.

As a rule, remove the core from the covers once a week and let it stand upright to air out while the covers are being washed.

It is recommended to order a spare cover so that the seat cushion can remain in use while the original cover is being washed

8. Washing Instructions/ Recycling

 The core and the covers are to be washed separately. For EasyCare+ cover, mild detergent can be used. **DO NOT USE CHLORINE BLEACH.** After washing, shake the core to remove excess water and let it stand to air drying. Do not use harsh cleaners such as scouring pads & heavy-duty grease removers, or solvents, such as acetone. To remove spots and stains, use standard house- hold cleansers and a soft cloth. To loosen heavy, dried-on soil, you may first need to saturate the spot and this can most often be solved by machine wash.

8.1 Cleaning and washing

The only need for maintenance of the cushion is cleaning:

1. Open the zipper at the rear and take the outer cover off.
2. Take out the core and place it for airing while the covers are washed. The inner core can also be rinsed under a tap or shower at 65 degrees or washed at gentle washing program in a washing machine at maximum 65 degrees for maximum 30 minutes. Shake the seat cushion a few times to shake the water of and let air-dry shortly if needed.
3. Follow washing instructions described on label attached to the bottom of the inner and outer cover .
4. The core is placed in the covers with the label facing down. For correct position of front and rear, see label.
5. Mount the outer cover. Make sure that the zipper faces rearwards and that the label faces down

Always follows the cleaning instruction label on cushions and covers.

Use washing bag for the covers - the Velcro may cause extra wearing during washing.

Male" part of Velcro can be cleaned with a steel file brush.

Core:	Netti Precision seat cushions
Wash	Rinsed by hand under cold water in the shower, max machine wash 65 °C at max 30 minutes without spin cycle
Desinfection, regular*	Virkon S*. Remove visible dirt, spray the dirty part of the surface, wait 1–2 minutes and wipe.
Drying	Air dry upright
Outer cover:	
Wash	Machine wash max 70 °C
Desinfection regular	Virkon S*. Remove visible dirt, spray the dirty part of the surface, wait 1–2 minutes and wipe.
Drying	Tumble dry max 80 °C

* Read instructions on core, on cushion and on package.

8.2 Reuse

Alu Rehab considers seat cushions to be an intimate and very personal medical accessory product. For recirculation, it is recommended to at least replace the outer cover and, if needed, the inner cover of the cushion.

It is mandatory to replace the cushion if the last user had infection - possibly with multi-resistant bacteria.

9. Tests & Warranty

9.1 Testing

Netti Precision seat cushions are tested and have been approved for both indoors and outdoors use. Netti seat cushions are CE marked and are manufactured to comply with the Medical Device Regulation (EU) 2017/745.

Maximun user load for Netti Precision seat cushions: 160 kg.

Netti wheelchairs with Netti Precision cushions have been tested by an accredited German test laboratory according to EN 12183. Netti wheelchairs with Netti cushions have also been crash tested at TASS International AS, Netherlands, and evaluated by accredited German test laboratory according to ISO 7176-19. Both are approved for being used as a seat in a car. Netti Precision seat cushions have been tested according to the following other standards:
Fire resistance: EN 1021-1 and 2.
Skin contact and cell contact: Cytotoxicity according to ISO 10993-5. The TPU does not contain added bisphenols or phthalates.

9.2 Warranty

The durability and longevity of the Netti Precision seat cushions depends on the design, selected hardnesses and stress applied. Compared to equivalent hardness with conventional foam cushions (PU and PUR) the Netti Precision seat cushions are superior. Therefore, there is a 2-year warranty on all Netti labelled precision seat cushions.

 Alu Rehab is not responsible for any damage resulting from inappropriate or incorrect installation and / or repairs, wear, maintenance and washing as well as deliberate damage not approved by Alu Rehab or produced by third parties. In such cases, this warranty shall be considered null and void.

This warranty is only valid when the user uses, maintains and handles the seat cushion as described in this instructions for use.

The warranty is only valid when the user has followed our recommendations as described in "Routines for inspection".

9.3 Warranty claim

If a product has developed a fault during the warranty period as result of a defect in design or manufacturing, you may forward a warranty claim.

- Claims are to be forwarded as soon as a defect is discovered and not later than 2 weeks after the defect is discovered.
- Claims are to be addressed to the sales agent of the wheelchair. Please note that sales documentation has to be filled in and signed correctly with serial number and eventual NeC number in order to document time and place of the purchase of the wheelchair.
- The sales agent and Alu Rehab are to decide whether a defect is covered by this warranty. The claimer will be informed about the decision as soon as possible.
- If the claim is accepted, the sales agent and Alu Rehab representative are to decide if the product will be repaired, replaced or if the customer is entitled to a reduced price.
- If a warranty claim is judged to be invalid - after careful inspection of the defect (defect due to wrong use and/ or lack of required maintenance) you are free to decide if you want to have the product repaired (if possible) at your expense, or if you want to purchase a new product.

Normal wear, incorrect use or incorrect handling is not a reason for claims.

9.4 Combining with other products

Use of the cushion with wheelchairs or accessories not manufactured or supplied by Alu Rehab is permitted only where compatibility and safe performance have been verified.

Transport in motor vehicles

Statements regarding crash testing and use as a seat in a motor vehicle apply only to validated wheelchair and cushion combinations and under the conditions specified by the manufacturer.

PRODUCT RESPONSIBILITY

Netti Precision seat cushions has been tested /risk evaluated by Alu Rehab.

Substitutions or alterations by third part suppliers to Netti seat cushions requires the risk evaluation and acceptance of the product responsibility and safety for use of the wheelchair from the manufacturer that is performing the substitution or alteration.

Information on product safety notices and product recalls are available at my-netti.com

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

9.5 The environment and waste disposal

Alu Rehab and its suppliers wish to protect the environment. This means:

- That we avoid using environmentally harmful substances and processes to the greatest extent possible.
- That Alu Rehabs products are ensured a long service life and a high degree of flexibility - to benefit the environment and economy.
- That all packaging can be recycled.
- That the seat cushion was designed to make recycling easier.
- Please contact your local recycling firm to get updated information which components and materials can be recycled and how they are to be separated.

A recycling manual for Netti wheelchairs can be downloaded at my-netti.com

For visually impaired people, manuals and catalogues can be downloaded at my-netti.com

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